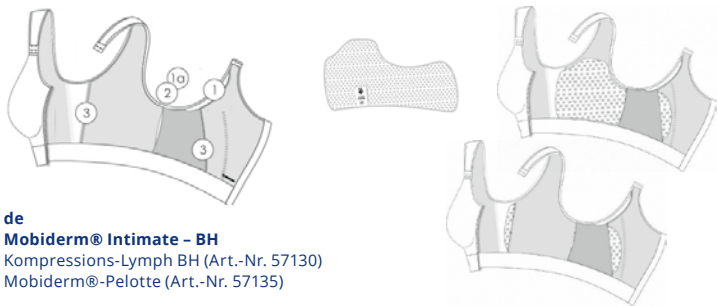
THUASNE®

SILIMA®

de	Gebrauchsanweisung - Mobiderm® Intimate – BH
en	Instructions for use - Mobiderm® Intimate – bra
fr	Consignes d'utilisation - Mobiderm® Intimate – SG
it	Istruzioni per l'uso - Mobiderm® Intimate – Breast
es	Instrucciones de utilización - Mobiderm® Intimate – pecho
pt	Instruções de uso - Mobiderm® Intimate soutien
nl	Gebruiksaanwijzing - Mobiderm® Intimate beha
da	Brugsanvisning - Mobiderm® Intimate bh
sv	Bruksanvisning - Mobiderm® Intimate bh
fi	Käyttöohje - Mobiderm® Intimate -rintaliivit
no	Bruksanvisning - Mobiderm® Intimate BH
el	Οδηγίες χρήσης - στηθόδεσμος Mobiderm® Intimate
cs	Návod k použití - Mobiderm® Intimate – podprsenka
pl	Sposób użytkowania - „Mobiderm® Intimate bra” biustonosz
et	Kasutusjuhend - Mobiderm® Intimate Rinnahoidja
lv	Lietošanas instrukcija - Mobiderm® Intimate — krūtīm
lt	Naudojimo instrukcija - „Mobiderm® Intimate – Breast”
sk	Návod na použitie - Mobiderm® Intimate – Breast
sl	Navodila za uporabo - Modrček Mobiderm® Intimate
hu	Használati utasítás - Mobiderm® Intimate Melltartó
ro	Instrucțiuni de utilizare - Sutien Mobiderm® Intimate
bg	Инструкция за употреба - Mobiderm® Intimate сутиен
hr	Upute za uporabu - Mobiderm® Intimate grudnjak
mk	Упатства за користење - Mobiderm® Intimate brus
sr	Uputstvo za upotrebu - Mobiderm® Intimate brus
tr	Kullanım talimatı: Mobiderm® Intimate göğüslük
ru	Инструкция по использованию - БРА Mobiderm® Intimate
ja	使用説明書 - Mobiderm® Intimate – Breast (モビダーム・インティメートブレスト)
ar	Mobiderm® Intimate – حمالة صدر - إرشادات الاستعمال
he	BH – מובידרם אינטימי - הוראות לשימוש
ko	독일어: 사용설명서 - Mobiderm® Intimate 브라
zh	使用说明书 - Mobiderm® Intimate 文胸

de

Mobiderm® Intimate – BH

Kompressions-Lymph BH (Art.-Nr. 57130)

Mobiderm®-Pelotte (Art.-Nr. 57135)

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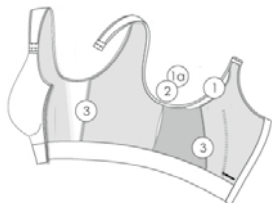
Mobiderm® pad (Art. No. 57135)

Indication and intended purpose

The compressive lymph bra is used for the compression and drainage of edemas in the area of the chest and thorax wall following mammary carcinoma.

The individually adjustable Mobiderm® pad achieves a local pressure increase on the tissue affected by edemas.

Therapeutic basis: The Mobiderm® pad consists of small cubes of 5×5mm which generate a pressure difference between the bearing surface of the foam cubes and the adjoining surroundings which results in a pulling effect among the treated subcutaneous tissue areas. Application for the uniform compression and support of the breast.

Controlled compression in the scar tissue area minimizes the formation of hematomas. Lymph outflows are drained and thus relieved. The healing process is supported.

The compressive lymph bra in different sizes consists of a highly elastic and skin-friendly outer material. The lining which takes up not only the breast form but also the pad consists of a soft and breathable tulle netting.

The Mobiderm® pad can be removed and is individually adaptable. It can be separately reordered as a spare.

The padded and triply adjustable straps, hooks and eyes as well as the front zipper provide for the bra's easy putting on and taking off and its adjustment.

The broad, elastic underbust band prevents rolling, pressure and abrasion points as well as constrictions. Form stabilization and immobilization of the post-surgical breast after any plastic breast surgery.

Indications

- Acute or chronic edemas and lymphoedemas of the breast and/or the breast wall
- Postoperative edemas or edemas after radio therapy (only on intact skin)
- Chronic infiltration after reconstruction, remaining chronic breast wall edemas

Contraindications

Allergic or inflammatory skin changes (e.g. swelling, reddening of the body area to be cared for). Use only on intact skin, not on infected or acutely inflammatory skin. Please consult with the treating physician regarding the indication and wearing mode of this orthopedic aid.

Size : XS-XL

Bust Girth: (please see Table)

Cup sizes: AA/A, B/C, D/E

	XS	S	M	L	XL
Int/ D	70 – 75	80 – 85	90 – 95	100 – 105	110 – 115
F/ E	85 – 90	95 – 100	105 – 110	115 – 120	125 – 130
I	1 – 2	3 – 4	5 – 6	7 – 8	9 – 10
GB/ US	32 – 34	36 – 38	40 – 42	44 – 46	48 – 50
AUS/ NZ	10 – 12	14 – 16	18 – 20	22 – 24	26 – 28

Putting the bra on

The bra is put on and closed by the front hooks and eyes and by the zipper. The straps are lengthwise adjustable in three steps. Please insert the pad as described in the drawing:

1) Pocket opening for epithesis

1a) Additional pocket opening for adjustment of the epithesis

2) Pocket opening for the Mobiderm® pad

3) Additional pocket engagement for adjustment of the Mobiderm® pad

- Insert the front part of the Mobiderm® pad into the pocket opening 2 and push it in the direction of the front part
- Use the front opening 3 for adjustment
- Push the rear part of the Mobiderm® pad into the rear pocket
- Use the rear opening for adjustment
- Please note: The pad is initially larger than the pockets. Once the bra is put on and stretched, the pad is entirely covered by the pocket.

Precautionary measures

Please follow the instructions by the physician or the orthopedic technician. In case of severe discomfort or unpleasant sensations, please contact your physician or orthopedic technician. Store at room temperature, preferably in its original packaging. Please follow the instructions by the physician or the orthopedic technician. This medical device is intended for the care of an individual female patient. In order to ensure its efficacy and for reasons of hygiene, this device may not be passed on to other female patients. Any serious incidents which occurred in connection with this product are to be reported to the manufacturer and the responsible authority of the member state in which the user and/or the patient are resident.

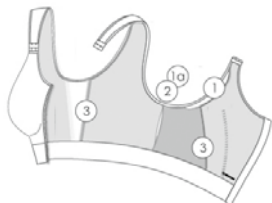
Washing instructions

30 degrees water temperature; no fabric softener (also see textile label)

Prior to washing, please close all closures!

Warranty

According to the statutory provisions, warranty is extended to material and processing if properly handled according to the instructions for care.

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Thuasne Scandinavia AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
Thuasne Deutschland GmbH