THUASNE®

Schulter-Fix Universal (SFU)

- de Gebrauchsanweisung - Schulterabduktionsorthese
- en Instructions for use - Shoulder abduction orthosis
- fr Consignes d'utilisation - Gilet d'immobilisation avec bras en abduction
- it Istruzioni per l'uso - Tutore per l'abduzione della spalla
- es Instrucciones de utilización - Órtesis de abducción de hombro
- pt Instruções de uso - Órtese para abdução do ombro
- nl Gebruiksaanwijzing - Schouderabductieorthese
- da Brugsanvisning - Skulderabduktionsortose
- sv Bruksanvisning - Axelsträckningsortos
- fi Käyttöohje - Olkaniveltukiside
- el Οδηγίες χρήσης - Νάρθηκας απαγωγής ώμου
- cs Návod k použití - Abdukční ortéza ramene
- pl Koszobó użytkowania - Orteza abdukcijna ramienia
- et Kasutusjuhend - Õla eemalhoideortos
- lv Lietošanas instrukcija - Pleca abdukcijas ortoze
- lt Naudojimo instrukcija - Peties abdukcinis įtvaras
- sk Návod na použitie - Abdukčná ortéza ramenného kĺbu
- sl Navodila za uporabo - Ortoza za ramenski abduktor
- hu Használati utasítás - Abdukciós válllortézis
- ro Instrucțiuni de utilizare - Orteză pentru abducția umărului
- bg Инструкция за употреба - Ортеза за абдукция на рамото
- hr Upute za uporabu - Abdukcijska ortoza za rame

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100	< 95 cm < 37 ½ inch
130	85 - 125 cm 33 ½ - 49 ½ inch

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¹Size · Taille · Taglia · Talla · Tamanho · Maat · Størrelse · Storlek · Koko · Μέγεθος · Velikost · Rozmiar · Suurus · izmērs · Dydis · Veľkosť · Velikost · Méret · Mărimē · Размер · Veličina

²Waist · Tour de taille · Circonferenza vita · Contorno de la cintura · Perímetro da cintura · Taille-omvang · Taljeomkreds · Midjemått · Vyötärön ympärys · Περίμετρος μέσης · Obvod pasu · Obwód talii · Talje ümbermõõt · vidukļa apkārtmērs · Liemens apimtis · Obvod pásu · Obseg pasu · Haskörfogat · circumferința talie · Обиколка на талията · Opseg struka

³Symmetric · Modèle unique, droite/gauche · Uguale per entrambi i lati · Simétrico · Lados iguais · Symmetrisch · Samme i begge sider · Symmetrisk · Sama molemilla puolilla · Ίδιο και για τις δύο πλευρές · Universální pro levou i pravou nohu · Obie strony jednakowe · Võrdsete külgedega · Abas puses vienādas · Dvipusis · Obojstranný model, pravý/lavý · Obe strani enaki · Kétoldalas, jobbos-balos modell · Laterale exacte · симетричен · Isto s obje strane

Schulterabduktionsorthese

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Shoulder abduction orthosis

Purpose

Schulter-Fix Universal is a shoulder joint orthosis for immobilising and relief with abduction position. The orthosis may only be used for shoulder treatment.

Mode of action

This shoulder orthosis holds and fixates the arm in place at the shoulder joint in the required abduction position. Due to the numerous options for adjustment, the orthosis can be individually adjusted to the arm to rest in the therapeutically desired position. Abduction angles of 0°, 30° and 45° may be set.

Indications

Reconstructions of the rotator cuff, shoulder luxations and distorsions, subcapital humerus fractures, SLAP lesions.

Contraindications

Larger skin injuries or diseases, extensive haematomas and severe swellings.

Side effects

When properly applied, no side effects are known to date.

Components

- (1) Torso belt
- (2) Wedge pad
- (3) Shoulder strap
- (4) Arm rest
- (5) Forearm strap
- (6) Wrist strap

Fitting instructions

The orthosis may be fitted on the right side as well as on the left.

Fitting the torso belt (1)

Place the torso belt around the body below the chest and close the Velcro fastening at the front.

Fitting the wedge pad (2) for abduction

Abduction angles of 45° and 30° may be set with the wedge pad. The appropriate side of the pad must face upward. Now, fasten the wedge pad with the open fastening area on the torso belt. Make sure that the fastening area is completely covered. Omitting the wedge pad results in an abduction angle of 0°.

Fitting the shoulder strap (3)

At an abduction angle of 30°, a shoulder strap is not absolutely necessary. At a 45° angle, it is advisable to use it. The shoulder strap is fastened on the sides of the wedge pad. For shortening the strap, please remove the Y-fasteners and use scissors to cut the strap to the desired length.

Fitting the forearm rest (4)

Position the forearm rest centred on the wedge pad under the arm to be fixated. Now, fasten the forearm strap (5) and the wrist strap (6).

Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Care instructions

Please use only mild detergents. Close Velcro fasteners before washing. After washing, carefully press out the orthosis (do not wring out) and allow to air-dry. Use a damp cloth to manually clean all paddings and wedges.

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Thuasne UK Ltd
Thuasne Deutschland GmbH