**THUASNE**

# Ligaflex® Action

fr	Attelle de stabilisation du poignet	5
en	Wrist stabilising splint	6
de	Orthese zur Stabilisierung des Handgelenks	7
nl	Stabilisatiebrace voor de pols	8
it	Tutore stabilizzatore per polso	10
es	Férula estabilizadora de la muñeca	11
pt	Tala de estabilização do punho	12
da	Stabiliserende skinne til tommelfingeren	13
fi	Stabiloiva rannetuki	15
sv	Stabiliseringsskena för handleden	16
el	Νάρθηκας σταθεροποίησης του καρπού	17
cs	Ortēza pro stabilizaci zápěstí	18
pl	Stabilizująca ortęza nadgarstka	19
lv	Plaukstas locītavas stabilizācijas saite	21
lt	Riešo stabilizavimo įtvaras	22
et	Randme stabiliseerimislahas	23
sl	Opornica za stabilizacijo zapestja	24
sk	Stabilizačná dlaha na zápästie	26
hu	Csuklórögzítő	27
bg	Шина за стабилизация на китка	28
ro	Atelă stabilizatoare pentru încheietura mâinii	29
ru	Стабилизирующая шина для запястья	31
hr	Udlaga za stabilizaciju ručnog zgloba	32
zh	腕部稳定夹板	33
ar	جبيرة لتثبيت الرسغ	37

	1	14 - 15 cm
	2	15 - 16 cm
	3	16 - 17 cm
	4	17 - 18 cm
	5	18 - 19 cm
	6	19 - 20 cm

<b>fr</b>	Tour de poignet
<b>en</b>	Wrist circumference
<b>de</b>	Handgelenkumfang
<b>nl</b>	Polsomtrek
<b>it</b>	Circonferenza del polso
<b>es</b>	Contorno de la muñeca
<b>pt</b>	Perímetro do punho
<b>da</b>	Håndleddets omkreds
<b>fi</b>	Ranteen ympärymitta
<b>sv</b>	Handledens omkrets
<b>el</b>	Περίμετρος καρπού
<b>cs</b>	Obvod zápěstí
<b>pl</b>	Obwód nadgarstka
<b>lv</b>	Plaukstas locītavas apkārtmērs
<b>it</b>	Riešo apimtis
<b>et</b>	Randme ümbermõõt
<b>sl</b>	Obseg zapestja
<b>sk</b>	Obvod zápästia
<b>hu</b>	Csukló kerülete
<b>bg</b>	Обиколка на китката
<b>ro</b>	Circumferința încheieturii
<b>ru</b>	Обхват запястья
<b>hr</b>	Obujam ručnog zgloba
<b>zh</b>	腕围
<b>ar</b>	محيط الرسغ

<b>fr</b>	Stabilisation
<b>en</b>	Stabilisation
<b>de</b>	Stabilisierung
<b>nl</b>	Stabilisatie
<b>it</b>	Stabilizzazione
<b>es</b>	Estabilización
<b>pt</b>	Estabilização
<b>da</b>	Stabilisering
<b>fi</b>	Stabiloi
<b>sv</b>	Stabilisering
<b>el</b>	Σταθεροποίηση
<b>cs</b>	Stabilizace
<b>pl</b>	Stabilizacja
<b>lv</b>	Stabilizēšana
<b>lt</b>	Stabilizavimas
<b>et</b>	Stabiliseerimine
<b>sl</b>	Stabiliziranje
<b>sk</b>	Stabilizácia
<b>hu</b>	Stabilizálás
<b>bg</b>	Стабилизация
<b>ro</b>	Stabilizare
<b>ru</b>	Стабилизация
<b>hr</b>	Stabilizacija
<b>zh</b>	稳定
<b>ar</b>	التثبيت

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## WRIST STABILISING SPLINT

### Description/Destination

This device is a wrist splint providing joint stabilisation. The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### Composition

Polyamide - polyester - cotton - elastane - polyurethane - aluminium - silicone.

### Properties/Mode of action

Stabilisation of the wrist via a rigid, malleable and anatomical palmar stay. Anatomically-shaped knit ensuring proprioception. Comfort zone at the ulnar head (silicone pad). Removable strap to adjust compression level.

### Indications

Wrist sprain.  
Wrist tendinopathies.  
Post-operative, post-traumatic residual pain and/or edema.  
Wrist osteoarthritis.

### Contraindications

Do not use the product if the diagnosis has not been confirmed.  
Do not apply the product in direct contact with broken skin.  
Do not use in the event of known allergy to any of the components.

### Precautions

Verify the product's integrity before every use.  
Do not use the device if it is damaged.  
Choose the appropriate size to fit the patient, referring to the size chart.  
It is recommended that a healthcare professional supervises the first application.  
Strictly comply with your healthcare professional's prescription and recommendations for use.  
In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.  
For hygiene and performance reasons, do not re-use the product for another patient.  
Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

### Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### Instructions for use/Application

If necessary, shape the palmar stay.  
Slip on the splint, making sure that the thumb is correctly positioned in the opening. Position the ulnar head in the hole of the silicone insert.  
If additional pressure is required, add the removable strap.  
Attach the strap onto the self-fastening surface on the inside of the splint, then wrap the strap around the wrist.  
Removal: To remove the splint, pull simultaneously the knit and the stay.

### Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Remove stay before washing. Replace in the original location before next use. Close the self-fastening tabs before washing. Machine washable at 30°C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

### Storage

Store at room temperature, preferably in the original packaging.

### Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2027401 (2021-04)