THUASNE®

Malleo Promaster

- de Gebrauchsanweisung - Sprunggelenkbandage mit verstärktem propriozeptiven Effekt
- en Instructions for use - Ankle brace with reinforced proprioceptive effect
- fr Consignes d'utilisation - Cheville à effet proprioceptif renforcé
- it Istruzioni per l'uso - Cavigliera con effetto proprioceptivo rinforzato
- es Instrucciones de utilización - Tobillera con efecto proprioceptivo reforzado
- pt Instruções de uso - Artelheira com efeito proprioceptivo reforçado
- nl Gebruiksaanwijzing - Enkelverband met versterkt proprioceptief effect
- da Brugsanvisning - Ankelbind med forstærket proprioceptiv virkning
- sv Bruksanvisning - Ankelledsförband med förstärkt proprioceptiv verkan
- fi Käyttöohje - Tukeva proprioseptiivinen nilkkatuki
- el Οδηγίες χρήσης - Επιστραγαλίδα με ενισχυμένη ιδιοδεκτικότητα
- cs Návod k použití - Kotníková ortéza se zesíleným účinkem propriocepce
- pl Sposób użytkowania - Orteza skokowo-stopowa ograniczająca supinację stopy
- et Kasutusjuhend - Tugevdatud propriotseptiivse toimega hüppeliigesetugi
- lv Lietošanas instrukcija - Pēdas locītavas ortoze ar stiprāku ietekmi propriocepcijai
- lt Naudojimo instrukcija - Čiurnos įtvaras su sustiprintu propriocepiniu poveikiu
- sk Návod na použitie - Členková ortéza so zosilneným proprioceptívnym účinkom
- sl Navodila za uporabo - Gleženjski povoj z okrepljenim proprioceptivnim učinkom
- hu Használati utasítás - Megerősített proprioceptív boka ortézis
- ro Instrucțiuni de utilizare - Glezniere cu efect proprioceptiv pronunțat
- bg Инструкция за употреба - Ортеза за глезен с подсилено проприоцептивно действие
- hr Upute za uporabu - Bandaža za gležanj s pojačanim proprioceptivnim učinkom

Größe ¹	Knöchelumfang ²	
	cm	inch
XS	17 - 19	6 ² / ₃ - 7 ¹ / ₂
S	19 - 21	7 ¹ / ₂ - 8 ¹ / ₃
M	21 - 23	8 ¹ / ₃ - 9
L	23 - 25	9 - 9 ³ / ₄
XL	25 - 27	9 ³ / ₄ - 10 ¹ / ₂
XXL	27 - 29	10 ¹ / ₂ - 11 ¹ / ₂

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¹Size · Taille · Taglia · Talla · Tamanho · Maat · Størrelse · Storlek · Koko · Μέγεθος · Velikost · Rozmiar · Suurus · izmērs · Dydis · Veľkosť · Velikost · Méret · Mărire · Размер · Veličina

²Ankle circumference · Tour de cheville · Circonferenza caviglia · Contorno del tobillo · Perímetro do tornozelo · Enkelomvang · Ankelomkreds · Benvidd · Nilkan ympärys · Περίμετρος σφυρού · Obvod kotníku · Obwód kostki · Hüppeliigeseümberrmõõt · Potītes apkārtnērs · Čiurnos apimtis · Obvod členku · Obseg gležnja · Bokakörfogat · Circumferința maleolă · Обиколка на глезена · Opseg gležnjeva

³Left or right · Droite ou gauche · Sinistro o destro · Izquierda o derecha · Esquerda ou direita · Links of rechts · Venstre eller højre · Vänster eller höger · Vasen tai oikea · Αριστερ. ή δεξ. · Vlevo nebo vpravo · Strona lewa lub prawa · Vasakul või paremal · Kreisā vai labā puse · Kairēje arba dešinėje · Vľavo alebo vpravo · Levo ali desno · Balos vagy jobbos · Stânga sau dreapta · лев или десен · Lijevo ili desno

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Ankle brace with reinforced proprioceptive effect**Purpose:**

Malleo Promaster and Malleo Promaster Aktiv are compression bandages with pads for treating ankle irritation. The bandages may only be used for ankle treatment.

Properties:

- Brace made of elastic knit
- 3D knit at the malleoli
- Malleolar pads
- Latex-free

Indications:

Post-traumatic, postoperative and chronic inflammation; Sprains; Joint effusions and oedema linked to arthritis and joint disease; Moderate joint instability

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

Make sure that the two malleoli are positioned in the 3D knit zones specially designed for this purpose. The ankle brace can be worn against the skin or on top of a well fitting sock.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Washing instructions:

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

**Composition:**

78 % polyamide, 14 % elastane, 5 % cotton, 2 % polyurethane, 1 % polyester

Explanation of the reinforced proprioceptive effect:

Orthoses in the Promaster range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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Thuasne Cervitex Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
Thuasne Scandinavia AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
Thuasne Deutschland GmbH