THUASNE®

Pero-med AFO III/II

de	Gebrauchsanweisung - Peroneusorthese
en	Instructions for use - Peroneal orthosis
fr	Consignes d'utilisation - Releveur de pied
it	Istruzioni per l'uso - Ortesi peroneale
es	Instrucciones de utilización - Órtesis para el levantamiento del pie
pt	Instruções de uso - Órtese para elevação do pé
nl	Gebruiksaanwijzing - Peroneusorthese
da	Brugsanvisning - Fodortose
sv	Bruksanvisning - Ankelfotortos
fi	Käyttöohje - Ortoosi jalkaterän noston avuksi
el	Οδηγίες χρήσης - Νάρθηκας έσω υποδήματος
cs	Návod k použití - Hlezenní ortéza
pl	Sposób użytkowania - Orteza na opadającą stopę
et	Kasutusjuhend - Hüppeliigeseortoos
lv	Lietošanas instrukcija - Krītošās pēdas ortoze
lt	Naudojimo instrukcija - Pėdos pakėlimo įtvėras
sk	Návod na použitie - Peroneálna ortéza
sl	Navodila za uporabo - Opornica za dvig padajočega stopala
hu	Használati utasítás - Peroneus-emelő
ro	Instrucțiuni de utilizare - Orteză pentru picior căzut
bg	Инструкция за употреба - Ортеза за разгъвача на ходилото
hr	Upute za uporabu - Ortoza za mišić za podizanje stopala

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Peroneal orthosis**Purpose**

Pero-med AFO III/II is a thermoplastically flexible foot orthosis for correction and/or pressure relief. The orthosis may only be used for foot treatment.

Indications

Mild to medium foot lifting weakness, mild peroneal pareses of the IIIrd and IInd degree without spasticity

Contraindications

Skin lesions or diseases in the affected area, especially with inflammatory symptoms. Sensation and blood circulation disturbances of the foot. Lymph flow disturbances, soft tissue swellings of unknown origin. Forefoot or heel ulcer.

Side effects

No side effects are as yet known for correct use.

Application instructions

- Open the calf Velcro strap.
- Insert the orthosis into the open shoe.
- Put the shoe on and position the lower leg in the orthosis
- Guide the calf strap round the calf under slight tension and close it.
- Close the shoe. The Pero-med AFO does only work in combination with a well-fitting shoe.

Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

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Thuasne Cervitex Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
Thuasne Scandinavia AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
Thuasne Deutschland GmbH