THUASNE®

Schulter-Fix Abdukt (SFA)

- de Gebrauchsanweisung - Schulterabduktionsorthese
- en Instructions for use - Shoulder abduction orthosis
- fr Consignes d'utilisation - Gilet d'immobilisation avec bras en abduction
- it Istruzioni per l'uso - Tutore per l'abduzione della spalla
- es Instrucciones de utilización - Órtesis de abducción de hombro
- pt Instruções de uso - Órtese para abdução do ombro
- nl Gebruiksaanwijzing - Schouderabductieorthese
- da Brugsanvisning - Skulderabduktionsortose
- sv Bruksanvisning - Axelsträckningsortos
- fi Käyttöohje - Olkaniveltukiside
- el Οδηγίες χρήσης - Νάρθηκας απαγωγής ώμου
- cs Návod k použití - Abdukční ortéza ramene
- pl Posób użytkowania - Orteza abdukcji ramienia
- et Kasutusjuhend - Õla eemalhoideortos
- lv Lietošanas instrukcija - Pleca abdukcijas ortoze
- lt Naudojimo instrukcija - Peties abdukcinis įtvaras
- sk Návod na použitie - Abdukčná ortéza ramenného kĺbu
- sl Navodila za uporabo - Ortoza za ramenski abduktor
- hu Használati utasítás - Abdukciós vállortézis
- ro Instrucțiuni de utilizare - Orteză pentru abducția umărului
- bg Инструкция за употреба - Ортеза за абдукция на рамото
- hr Upute za uporabu - Abdukcijska ortoza za rame

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100	< 95 cm < 37 ½ inch
130	85 - 125 cm 33 ½ - 49 ½ inch

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





¹Size · Taille · Taglia · Talla · Tamanho · Maat · Størrelse · Storlek · Koko · Μέγεθος · Velikost · Rozmiar · Suurus · izmērs · Dydis · Veľkosť · Velikost · Méret · Mărimē · Размер · Veličina

²Waist · Tour de taille · Circonferenza vita · Contorno de la cintura · Perímetro da cintura · Taille-omvang · Taljeomkreds · Midjemått · Vyötärön ympärys · Περίμετρος μέσης · Obvod pasu · Obwód talii · Talje ümbermõõt · vidukļa apkārtmērs · Liemens apimtis · Obvod pásu · Obseg pasu · Haskörfogat · circumferința talie · Обиколка на талията · Opseg struka

³Symmetric · Modèle unique, droite/gauche · Uguale per entrambi i lati · Simétrico · Lados iguais · Symmetrisch · Samme i begge sider · Symmetrisk · Sama molemilla puolilla · Ίδιο και για τις δύο πλευρές · Universální pro levou i pravou nohu · Obie strony jednakowe · Võrdsete külgedega · Abas puses vienādas · Dvipusis · Obojstranný model, pravý/lavý · Obe strani enaki · Kétoldalas, jobbos-balos modell · Laterale exacte · симетричен · Isto s obje strane

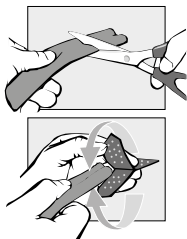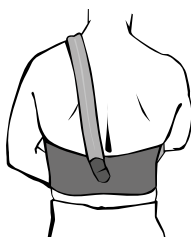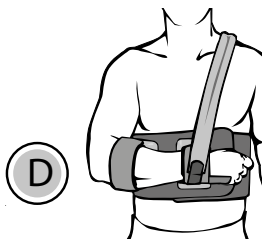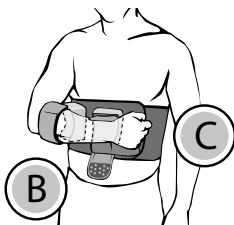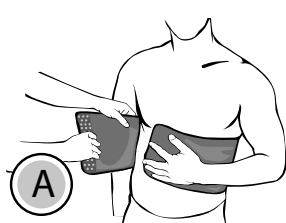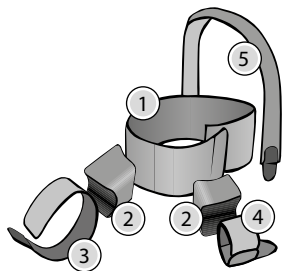Schulterabduktionsorthese

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Shoulder abduction orthosis

Purpose

Schulter-Fix Abdukt is a shoulder joint orthosis for immobilising and relief with abduction position. The orthosis may only be used for shoulder treatment.

Mode of action

The Schulter-Fix Abdukt orthosis is used for postoperative fixation of the arm and shoulder. It is to secure surgery results and reduce possible pain.

Indications

Immobilisation after shoulder joint surgery, shoulder joint diseases, bursitis, upper arm fractures near the joint.

Contraindications

Larger skin injuries or diseases, extensive haematomas and severe swellings.

Side effects

When properly applied, no side effects are known to date.

Components

- (1) Torso belt
- (2) Foam blocks
- (3) Forearm fixation cuff
- (4) Wrist cuff with hand support
- (5) Arm carrying strap

Fitting instructions**A** Adjusting – torso belt (1)

For treatment of the right arm, close the torso belt to the left. For treatment of the left arm, please proceed in reverse order. Make sure that the torso belt does not fit too loosely.

B Foam blocks (2)

In a next step, please hold the arm to be fixated in the desired position. Now, use Velcro fasteners to fasten the large foam block in the proper position close to the elbow onto the bandage part. Then fasten the second foam block directly underneath the wrist on the foam part.

C Forearm fixation cuffs (3+4)

The orthosis has two cuffs to fasten the arm. In open position, these are fastened in the curvature of the foam blocks such that the Velcro side faces downward. Subsequently, put the arm in and firmly close both cuffs. The hand support is fastened with Velcro strips in the wrist cuff (4) and provides relief support for the hand.

D Arm carrying strap (5)

Final secure fixation of the arm is achieved by applying the carrying strap. Use Velcro fasteners to fasten the carrying strap to the wrist cuff and then pull the strap diagonally past the neck. Then fasten it centred in the area of the back. The strap length may be shortened to a suitable length (see illustration).

Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Care instructions

Please use only mild detergents. Close Velcro fasteners before washing. After washing, carefully press out the orthosis (do not wring out) and allow to air-dry. Use a damp cloth to manually clean all paddings and wedges.

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Thuasne UK Ltd
Thuasne Deutschland GmbH