

LYMPHATREX Revolution

LEGS


1- Anatomical markers



Select the anatomical markers required according to the garment model chosen.

Position the markers on the outer side of the leg.

- (A) At the head of the metatarsals bones
- (Y) Between the instep and the tuberosity of the calcaneus
- (B) At the narrowest point of the ankle
- (B1) At the start of the calf muscles
- (C) At the widest point of the calf
- (D) At the head of the fibula (3 - 4 cm under the patella)
- (E) In the middle of the patella
- (F) Equal distant point between G and E
- (G) For thigh-high: is located at the desired fitting length
For thigh high with hip cover option: at the gluteal fold

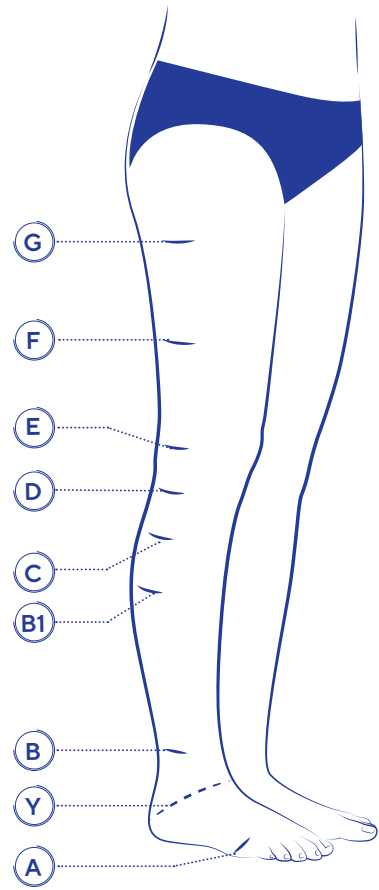


Important - do not forget to measure and indicate foot lengths (inside/outside).

2- Circumferences measurement



The measuring tape should be pulled firmly, but not so tightly that it indents the skin. Pressure should be adapted to the patient tolerance.

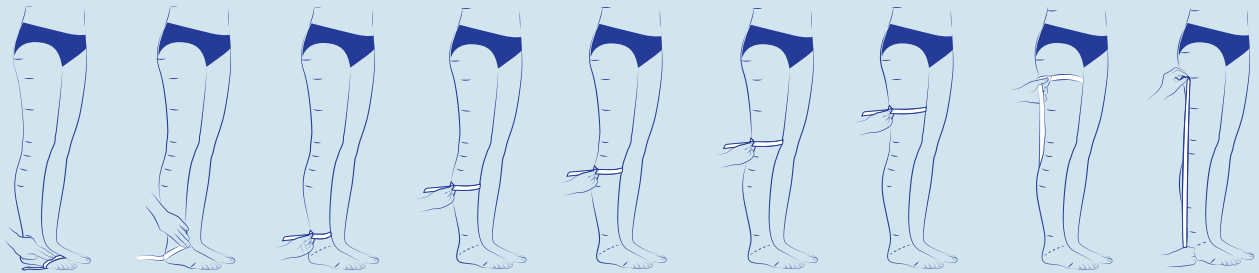


Leg lengths from the floor: cB, cB1, cC, cD, cE, cF, cG.


3- Lengths measurement



Take the measurements on the outer side of the leg with the measuring tape placed directly on the skin.

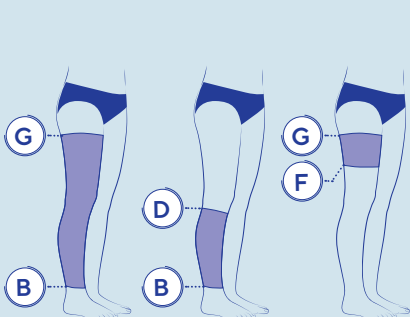

Leg sleeves: fill in the circumferences from cB to cG and lengths from lG to lB according to the leg sleeve model.

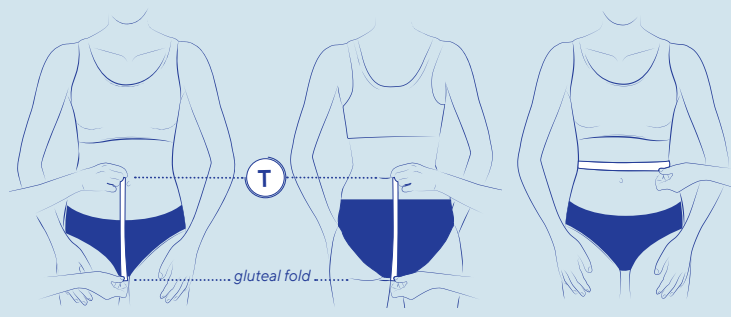


Front lGT length: take lGT from gluteal fold transfer to the desired length.

Back lGT length: take lGT from gluteal fold to the desired length.

cT

Hip cover option: add the measurements for the cT circumference and the front and back lGT lengths.

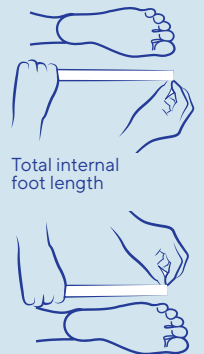

Total internal foot length

Total external foot length

Closed toe: total foot length
Open toe: desired garment length