MOBIDERM® autofit


fr	Vêtements ajustables mobilisateurs et compressifs pour les œdèmes	8
en	Mobilizing and compressive adjustable garments for edemas	8
de	Verstellbare mobilisierende Kompressionskleidung zur Behandlung von Ödemen	9
nl	Verstelbare mobilisatie- compressiekleding voor oedemen	9
it	Indumenti regolabili di mobilizzazione e compressione per gli edemi	10
es	Prendas movilizadoras y compresivas ajustables para edemas	10
pt	Roupas de mobilização ajustáveis e de compressão para os edemas	11
da	Mobiliserende, justerbare og kompressive beklædningsgenstande til ødemer	11
fi	Säädettävät mobilisoivat kompressiovaatteet turvotukseen	12
sv	Justerbara mobiliserande kompressionsplagg för ödem	12
el	Ρυθμιζόμενα ενδύματα κινητοποιήσης και προϊόντα συμπίεσης για οιδήματα	13
cs	Nastavitelné kompresní mobilizační prádlo pro léčbu edémů	14
pl	Regulowana odzież mobilizująca i uciskowa na obrzęki	14
lv	Regulējami mobilizējoši un kompresijas izstrādājumi tūskām	15
lt	Poodinę cirkuliaciją skatinantys ir kompresiniai reguliuojami drabužiai ištinusioms vietoms	15
et	Nahaaluseid kudesid mobiliseerivad kohandatavad riideesemed ödeemide puhuks	16
sl	Prilagodljiva mobilizacijska oblačila in komprese za edeme	16
sk	Nastaviteľné mobilizačné a kompresívne návleky na liečbu lymfedému	17
hu	Állítható mobilizáló és kompressziós ödémacsökkentő öltözet	17
bg	Регулируеми мобилизиращи и компресивни дрехи за отоци	18
ro	Îmbrăcămintele de mobilizare și compresie ajustabilă pentru edeme	18
ru	Регулируемая мобилизующая и компрессионная одежда для снятия отеков	19
hr	Prilagodljivi odjevni predmeti za imobilizaciju i kompresiju kod edema	19
zh	用于水肿治疗的可调节活血及压力治疗衣	20
ja	浮腫治療用の調節可能な圧迫・弾性衣服	20
ko	절 가능한 부종용 압박 및 모빌라이징 가먼트	21
ar	أغطية مُحركة للوذمات قابلة للتعديل وضاغطة	22

fr	Chaussette	nl	Kous	pt	Meia	sv	Knästrumpa	pl	Skarpetka	et	Sokk	hu	Zokni	ru	Носок	ja	靴下
en	Below-knee	it	Calza corta	da	Strømpe	el	Κάλτσα	lv	Īsā zeķe	sl	Nogavica	bg	Чопан	hr	Čarapa	ko	양말
de	Wadenstrumpf	es	Calcetín	fi	Sukka	cs	Podkolenka	lt	Puskojinė	sk	Lýtkový návlek	ro	Șosetă	zh	袜子	ar	جوارب

	cB (cm)	cC (cm)	cD (cm)	ØD (cm)			
1	16 – 24	24 – 35	25 – 36	S 38 – 40	N 40 – 42	L 42 – 44	XL 44 – 46
2	20 – 28	32 – 43	32 – 43				
3	24 – 32	40 – 51	39 – 50				
4	28 – 36	48 – 59	46 – 57				

fr	Bas-cuisse	nl	Dijkous	pt	Meia até à coxa	sv	Lårstrumpa	pl	Pończocha	et	Reieosa	hu	Combfix	ru	Чуллок	ja	ロングストッキング
en	Thigh-high	it	Calza alla coscia	da	Lårstrømpe	el	Κάλτσα ριζομπριου	lv	Garā zeķe	sl	Nadkolenka	bg	Чорап за бедрото	hr	Natkoljenka	ko	허벅지 스타킹
de	Oberschenkelstrumpf	es	Media-muslo	fi	Reisisukka	cs	Stehenní punčocha	lt	Kojinė iki klubo	sk	Stehenný návlek	ro	Ciorapi	zh	长筒袜	ar	جوارب فخذی

	cB (cm)	cC (cm)	cG (cm)	ℓG (cm)			
1	16 – 24	24 – 35	46 – 57	S 67 – 72	N 72 – 77	L 77 – 82	XL 82 – 87
2	20 – 28	32 – 43	53 – 65				
3	24 – 32	40 – 51	61 – 72				
4	28 – 36	48 – 59	68 – 80				

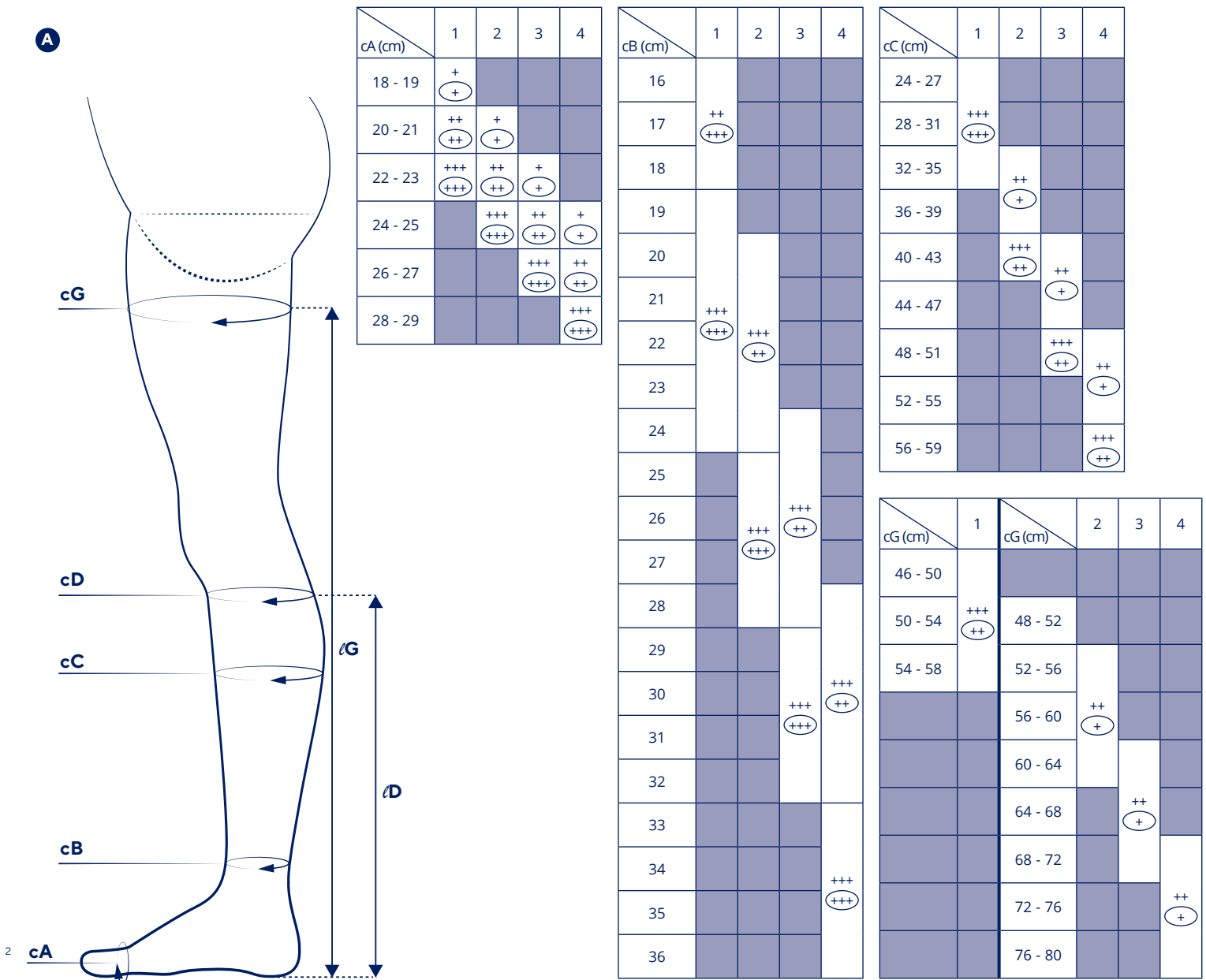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Conservser cette notice.

en

MOBILIZING AND COMPRESSIVE ADJUSTABLE GARMENTS FOR EDEMAS

Description/Destination

Mobiderm® Autofit mobilizing and compressive garments are intended for the treatment of edema (e.g. lymphedema) of the upper or lower limbs.

Target population: adult and children patients suffering from the indicated pathologies and compatible with the sizing chart.

This range is available in several models:

- Armsleeve with mitten
- Thigh-knee (bilateral)
- Below-knee (bilateral).

The garments are composed of:

- on the internal layer, Mobiderm® technology consisting of foam blocks encased between 2 sheets of non-woven fabric;
- on the external layer, soft, elastic beige fabric and hook-and-loop fasteners, allowing the garment to be adjusted to limb volume.

Two sizes of foam block are used depending on the body area treated: small blocks (5 x 5 mm) on the hand and foot, big blocks (15 x 15 mm) on the leg and arm.

Composition

Textile components: polyamide - polyester - cotton - polyurethane - elastane - acrylic adhesive mass - silicone (thigh-high and below-knee). Foam components: polyurethane (large blocks) - polyethylene (small blocks).

Properties/Mode of action

Mobiderm® Autofit garments provide:

- compression, by applying pressure to the limb, thus promoting lymphatic circulation and helping to maintain or reduce the volume of the edema;
- a subcutaneous drainage via the Mobiderm® technology consisting of foam blocks positioned randomly between two sheets of non-woven material. In contact with the skin, the blocks provide a pressure differential between the contact area beneath them and their periphery, creating multi-directional circulation channels for the lymph under the skin. The lymph travels from the areas pressed by the blocks to the free areas. Efficient lymph drainage and circulation are thus promoted. **C**

The garments are designed to be worn day and night.

Indications

Acute or chronic edema management (e.g: lymphedema during the intensive and/or maintenance phases).

Contraindications

Do not use in case of major dermatological conditions on an affected limb.

Do not use in the event of known allergy to any of the components.

Do not use in case of severe peripheral neuropathy of the affected limb.

Do not use in the event of septic thrombosis.

Do not use in the event of phlegmasia cerulea dolens (painful blue edema with arterial compression) of the affected limb.

Specific to the lower limbs:

Do not use in case of lower limb peripheral arterial disease (PAD) with ankle-brachial index (ABI) < 0.6.

Do not use in the event of decompensated heart failure.

Do not use in the event of advanced diabetic microangiopathy.

Do not use in case of extra-anatomical bypass of the affected limb.

Specific to the upper limbs:

Do not use in case of brachial plexus condition.

Do not use in case of vasculitis of the extremities.

Precautions

Strictly comply with your healthcare professional's prescription and recommendations for use.

The risk/benefit ratio and the adequate level of pressure must be regularly re-assessed by a healthcare professional in the event of:

- PAD with ABPI between 0.6 and 0.9
- Advanced peripheral neuropathy
- Weeping or eczematous dermatitis.

In the event of discomfort, significant restriction, pain, variation in limb volume, deterioration of skin condition, infection, unusual sensations, change in the colour of the extremities, or changes in the performance of the product, remove the product and seek the advice of a healthcare professional.

Do not apply the product directly to an open wound without a dressing.

During the lymphedema maintenance phase, the device must be worn at night as a replacement for the daytime compression garment. It must be kept on all night. It can be worn during the day when resting.

The application of this device can induce the appearance marks on the skin. These marks are benign and temporary; do not stop your treatment without medical advice.

For fragile skin, it is recommended to apply a cotton bandage to the limb before putting on the garment.

Due to the constant evolution of the pathology, regular follow-up with a healthcare professional is recommended to check that the compression device is still suitable for the pathology.

The use of a medical device by a child should be done under the supervision of an adult or a healthcare professional.

Do not use when swimming.

Do not use products on the skin (creams, ointments, oils, gels, patches...) before applying the device, it could damage the product.

Take care not to damage the device when fitting it, particularly with your nails.

In the event of associated advanced venous disease or venous ulcer, seek the advice of a healthcare professional for appropriate compression treatment on the entire leg.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity or cutaneous dryness.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Choose the appropriate size to fit the patient, referring to the size chart.

We recommend washing your new garment before use.

Verify the product's integrity before every use.

The Mobiderm® Autofit garment can be put on next to the skin like a garment or over a cotton bandage for fragile skin.

Donning of below-knee and thigh-high:

Open all the hook-and-loop fasteners on one side of the garment **⓪**.

Insert the foot into the garment **⓪** then slide the garment carefully along the limb to below the knee (for below-knee model) **⓪** or to below the gluteal fold (for thigh-high model).

Avoid pulling on the garment more than is necessary.

Close the hook-and-loop fasteners beginning at the foot **⓪** and working up to the knee or the gluteal fold **⓪**.

Proceed in several stages until the adjustment recommended by your healthcare professional is achieved (see "Adjustment of hook-and-loop fasteners" paragraph).

Once the garment is fully in place, massage the limb in order to ensure a homogeneous distribution of the foam blocks **⓪**.

Donning of armsleeve with mitten:

Open all the hook-and-loop fasteners **⓪**.

Insert the hand into the garment **⓪** then slide the garment carefully along the limb up to the armpit **⓪**.

Avoid pulling on the garment more than is necessary.

Close the hook-and-loop fasteners beginning at the hand **⓪** and working up to below the armpit **⓪**.

Proceed in several stages until the adjustment recommended by your healthcare professional is achieved (see "Adjustment of hook-and-loop fasteners" paragraph).

Once the garment is fully in place, massage the limb in order to ensure a homogeneous distribution of the foam blocks **⓪**.

Adjustment of hook-and-loop fasteners **A ⓪** :

Mobiderm® Autofit garments display tightening indications (+/++/+++ /++++) at several points.

These allow the garment to be adjusted to your morphology depending on the size of the garment that has been issued to you.

In the **maintenance** (lymphedema) phase, refer to the uncircled + lines. The +/++/+++ /++++ symbols are used as markers to adjust the hook-and-loop fasteners.

In the **intensive** (lymphedema) phase, refer to the circled + lines. The **(+)** **(++)** **(+++)** **(++++)** symbols are used as markers to adjust the hook-and-loop fasteners.

To easily find the recommended tightening according to the phase, refer to the various circumference tightening tables **A p.2** and **B p.4**.

Recommendations for adjustment of the hook-and-loop fasteners are also given at the start of the instruction leaflet for informational purposes.

In all cases refer to the recommendations of the healthcare professional who prescribed the product.

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©Thuasne - 2055001 (2024-06)
THUASNE UK Ltd