
THUASNE®



MOBIDERM

fr	Vêtements mobilisateurs pour les œdèmes	8
en	Mobilizing garments for edema	10
de	Mobilisierende Kleidung zur Behandlung von Ödemen	12
nl	Kledingmobilisators voor oedemen	14
it	Indumenti di mobilizzazione per edemi	16
es	Prendas movilizadoras para edemas	18
pt	Roupas de mobilização para os edemas	20
da	Mobiliserende beklædningsgenstande til ødemer	22
fi	Mobilisoivat vaatteet turvotukseen	24
sv	Mobiliserande plagg för ödem	26
el	Ενδύματα κινητοποιήσεως για τα οίδημα	28
cs	Mobilizační prádlo pro léčbu edémů	31
pl	Odzież mobilizująca na obrzęki	33
lv	Mobilizējoši apģērbi tūskām	35
lt	Poodinę cirkuliaciją skatinantys drabužiai ištinusioms vietoms	37
et	Nahaaluseid kudesid mobiliseerivad riideesemed ödeemide puhuks	39
sl	Mobilizacijska oblačila za edeme	41
sk	Mobilizačné návleky na liečbu lymfedému	43
hu	Ödémacsökkentő mobilizáló öltözet	45
bg	Мобилизиращи дрехи за отоци	47
ro	Îmbrăcămintele de mobilizare pentru edeme	49
ru	Одежда мобилизующая для лечения отеков	51
hr	Odjevni predmeti za imobilizaciju za liječenje edema	53
zh	用于水肿治疗的 活动治疗衣	55
ja	浮腫治療用の弾性衣服	57
ko	부종 모빌라이징 가먼트	59
ar	أغطية مُحركة للوذمات	63

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - 使用方法 - 사용설명서 - إرشادات الاستعمال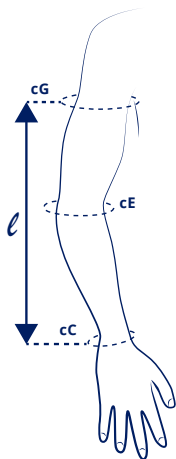

	cC (cm)	cE (cm)	cG (cm)	ℓ (cm)	
1	14 – 16	22 – 25	25 – 29	N 40 – 45	L 45 – 50
1+		24 – 27	28 – 32		
2	16 – 18	25 – 28	29 – 33		
2+		27 – 30	31 – 35		
3	18 – 20	28 – 31	32 – 36		
3+		30 – 33	35 – 39		
4	20 – 22	31 – 34	35 – 39		
4+		33 – 36	38 – 42		
5	22 – 24	34 – 37	38 – 42		
5+		36 – 39	41 – 45		

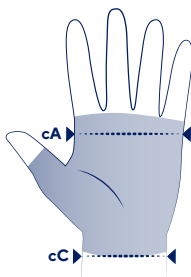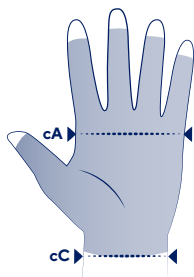	cC (cm)	cA (cm)
1	14 – 16	15 – 17
2	16 – 18	17 – 19
3	17 – 19	19 – 21
4	19 – 21	21 – 23
5	20 – 22	23 – 25
6	21 – 23	25 – 27

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MOBILIZING GARMENTS FOR EDEMA

Description/Destination

Mobiderm mobilizing garments are intended for the treatment of lymphedemas of the upper or lower limbs.

Target population: adult and children patients suffering from the indicated pathologies and compatible with the sizing chart.

This range is available in several models:

Mobiderm Standard:

- Armsleeve (bilateral)
- Mitten (bilateral)
- Glove

Mobiderm made-to-measure patient (for morphologies outside the size range):

- Armsleeve
- Armsleeve with mitten without thumb
- Armsleeve with mitten and thumb
- Thigh-high
- Below-knee

The garments are composed of:

- on the internal layer, Mobiderm technology consisting of foam blocks encased between 2 sheets of non-woven fabric;
- on the external surface, of beige-colored, soft, elastic material.

Two sizes of foam block are used depending on the models and the body area treated. The models contain either a single block size (5 x 5 mm or 15 x 15 mm) or both block sizes.

Composition

Textile components: polyamide - polyester - elastane - acrylic adhesive mass.

Foam components: polyurethane (large blocks) - polyethylene (small blocks).

Properties/Mode of action

Mobiderm garments provide:

- compression by applying pressure to the limb, thus promoting the circulation of lymph and helping to maintain or reduce swelling;
- a subcutaneous drainage via the Mobiderm technology consisting of foam blocks positioned randomly between two sheets of non-woven material. In contact with the skin, the blocks provide a pressure differential between the contact area beneath them and their periphery, creating multi-directional circulation channels for the lymph under the skin. The lymph travels

from the areas pressed by the blocks towards the free areas. Efficient lymph drainage and circulation are thus promoted. Ⓐ

The garments are designed to be worn day and night.

Indications

Acute or chronic edema management (e.g: lymphedema during the maintenance phase).

Contraindications

Do not use in case of major dermatological conditions on an affected limb.

Do not use in the event of known allergy to any of the components.

Do not use in case of severe peripheral neuropathy of the affected limb.

Do not use in the event of septic thrombosis.

Do not use in the event of phlegmasia cerulea dolens (painful blue phlebitis with arterial compression) of an affected limb.

Specific to the lower limbs:

Do not use in case of lower limb peripheral arterial disease (PAD) with ankle-brachial index (ABI) < 0.6.

Do not use in the event of decompensated heart failure.

Do not use in the event of advanced diabetic microangiopathy.

Do not use in case of extra-anatomical bypass of the affected limb.

Specific to the upper limbs:

Do not use in case of brachial plexus condition.

Do not use in case of vasculitis of the extremities.

Precautions

Strictly comply with your healthcare professional's prescription and recommendations for use.

The risk/benefit ratio and the adequate level of pressure must be regularly re-assessed by a healthcare professional in the event of:

- PAD with ABPI between 0.6 and 0.9
- Advanced peripheral neuropathy
- Weeping or eczematous dermatitis.

In the event of discomfort, significant restriction, pain, variation in limb volume, deterioration of skin condition, infection, unusual sensations, change in the colour of the extremities, or changes in the performance of the product, remove the product and seek the advice of a healthcare professional.

Do not apply the product directly to an open wound without a dressing.

During the lymphedema maintenance phase, the device must be worn at night in alternation with the compression garment worn during the day. It must be kept on all night. It can be worn during the day during rest phases.

Application of this device can cause the appearance of marks on the skin. These marks are benign and temporary; do not stop your treatment without medical advice.

For fragile skin, it is recommended to apply a cotton bandage to the limb before putting on the garment.

Due to the constant evolution of the pathology, regular follow-up with a healthcare professional is recommended to check that the compression device is still suitable for the pathology.

The use of a medical device by a child should be done under the supervision of an adult or a healthcare professional.

Do not use when swimming.

Do not use products on the skin (creams, ointments, oils, gels, patches...) before applying the device, it could damage the product.

Take care not to damage the device when fitting it, particularly with your nails.

In the event of associated advanced venous disease or venous ulcer, seek the advice of a healthcare professional for appropriate compression treatment on the entire leg. For hygiene, security and performance reasons, do not re-use the product for another patient.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity or cutaneous dryness.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Choose the appropriate size to fit the patient, referring to the size chart.

- We recommend washing your new garment before use.
- Verify the product's integrity before every use.
- The Mobiderm garment can be put on next to the skin like a garment or over a cotton bandage for fragile skin.

Donning of armsleeve and armsleeve with mitten ①:

- Insert your hand into the garment and turn the garment back on itself up to the opening provided for the hand ①.
- Insert your hand into the garment ② then slide the garment carefully along the limb up to the armpit ③.
- Avoid pulling on the garment more than is necessary.
- Once the garment is fully in place, massage the limb in order to ensure a homogeneous distribution of the foam blocks ④.

Donning of mitten and glove ⑤:

- Ensure that the foam blocks are positioned on the back of the hand ⑤.
- The mitten is bilateral: to change the mitten from right to left or from left to right, simply turn it inside out ⑥.
- Insert your hand into the garment then slide the garment carefully up to the wrist ⑦.
- Avoid pulling on the garment more than is necessary.
- When combined with an armsleeve, position the mitten or the glove above the end of the armsleeve at the wrist ⑧.
- Once the garment is fully in place, massage the limb in order to ensure a homogeneous distribution of the foam blocks ⑨.

Donning of below-knee and thigh-high ⑩:

- Insert your hand inside the garment and turn the garment back on itself up to the heel ⑩.
- Insert the foot into the garment until the heel is correctly positioned ⑪.
- Slide the garment carefully along the limb to below the knee (for below-knee model) ⑫ or to below the gluteal fold (for thigh-high model).
- Avoid pulling on the garment more than is necessary.
- Once the garment is fully in place, massage the limb in order to ensure a homogeneous distribution of the foam blocks ⑬.

Care/Maintenance

To ensure your Mobiderm garments last as long as possible and to prevent infection, it is important to wash them every day, or after each use.

Hand wash: Soak the device in cold soapy water, rub with a clean sponge and squeeze out excess water.

Air dry, away from direct heat (e.g. radiator, sun, etc.). Do not tumble-dry. Do not iron.

In a washing machine: Use the wool cycle at a cool temperature. Use a laundry detergent without fabric softener. Do not use chlorine-based detergents or harsh products.

Place the garment in a mesh laundry bag to prevent damage by other garments (particularly with self-fastening tabs or zippers). Squeeze out by hand.

Air dry, away from direct heat (e.g. radiator, sun, etc.). Do not tumble-dry. Do not iron.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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Come indossare il mezzoguento e il guanto ©:
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186