**THUASNE®**

# LigaStep AIR Walker

<b>fr</b> Botte de marche à montants fixes et chausson gonflable, version courte ou version longue.....	3
<b>en</b> Walking boot with rigid side uprights and inflatable liner - tall or short version.....	3
<b>de</b> Unterschenkel-Fußorthese mit Seitenstützen und Innenschuh mit aufblasbarer Luftkammer, kurzes und langes Modell.....	3
<b>nl</b> Walker met gefixeerde staanders en opblaasbare binnenschoen, korte uitvoering of lange uitvoering.....	4
<b>it</b> Stivale da deambulazione con scocca fissa e imbottitura gonfiabile, versione corta o versione lunga.....	4
<b>es</b> Bota de caminar con montantes fijos y botín hinchable, versión corta o versión larga .....	5
<b>pt</b> Bota de caminhada com suportes fixos e sapato insuflável, versão curta ou versão longa.....	5
<b>da</b> Støttestøvle med faste stivere og oppustelig overtræksko, i kort eller lang udgave.....	6
<b>fi</b> Kokonaan jäykkärunkoinen kävelysaapas puhallettavalla sisäosalla, lyhyt tai pitkä malli.....	6
<b>sv</b> Fotstöd med fast skal och uppblåsbar toffel, kort eller lång version.....	7
<b>el</b> Νάρθηκας ποδοκυνηγικής με σταθερές ράβδους και φουσκωτή κάλτσα, κοντή έκδοση ή υψηλή έκδοση .....	7
<b>cs</b> Fixační ortéza s pevnými výztuhami a nafukovací pěnovou vložkou, krátká nebo dlouhá verze.....	8
<b>pl</b> But ortopedyczny ze stałymi wzmocnieniami i z pompowanym miękkim butem, wersja krótka lub wersja dłuższa .....	8
<b>lv</b> Pastaigu zābaki ar fiksētiem balstiņiem un piepūšamu oderi, īsa vai gara versija .....	8
<b>lt</b> Eiti padedantis įtvaras fiksuoja statramais ir pripučiamu įdėklu, ilgas arba trumpas .....	9
<b>et</b> Jäigal korpusje ja pneumaatilise sussiga tugisaabas, lühike ja piklik variant.....	9
<b>sl</b> Škorenj s fiksirima opornikoma in napihljivo nogavico v kratki ali daljši različici .....	10
<b>sk</b> Členková dlahá s pevnými postrannými dlhami a nafukovacou vložkou, k rátka alebo dlhá verzia .....	10
<b>hu</b> Járótalpas bokarögzítő rögzített merevítővel és felfújható betéttel, rövid vagy hosszú kivitelben ... 10	10
<b>bg</b> Ортеза за ходене тип „ботуш“ с фиксирани опори и надуваш се чорап, къс или дълъг модел.....	11
<b>ro</b> Cizmă cu montanți fixi și șoson gonflabil, versiunea scurtă sau lungă .....	11
<b>ru</b> Сапог для ходьбы с фиксированными стойками и надувным сапожком, короткая или длинная версия.....	12
<b>hr</b> Čizma s čvrstim kalupom i čarapom koja se može napuhati, kratka ili duga verzija .....	12
<b>zh</b> 长款或短款助行靴 , 带固定护腿束带和充气衬板 .....	13
<b>ar</b> حذاء لالمشي ذو دعامات جانبية صلبة وبطانة اسفنجية قبلة للنفخ نسخة قصيرة او نسخة طويلة .....	13

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegółowe użyczkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - ارشادات الاستعمال

<b>XS</b>	< 35
<b>S</b>	35 - 38
<b>M</b>	39 - 43
<b>L</b>	44 - 47
<b>XL</b>	> 47

© Thuasne - 6172601 (2023-09)  
© Studio Caterin

THUASNE  
120, rue Marius Aufan  
92300 Levallois-Perret  
France

[www.thuasne.com](http://www.thuasne.com)

	<b>XS</b>	< 35
	<b>S</b>	35 - 38
	<b>M</b>	39 - 43
	<b>L</b>	44 - 47
	<b>XL</b>	> 47

<b>fr</b>	Taille - Pointure européenne
<b>en</b>	Size - European shoe size
<b>de</b>	Größe - Europäische Schuhgröße
<b>nl</b>	Maat - Europese schoenmaat
<b>it</b>	Taglia - Numero di scarpa europeo
<b>es</b>	Talla - Número de pie europeo
<b>pt</b>	Tamanho - Número do sapato europeu
<b>da</b>	Størrelse - Europæisk størrelse
<b>fi</b>	Koko - Kengän koko (EU)
<b>sv</b>	Storlek - Europeisk storlek
<b>el</b>	Μέγεθος - Ευρωπαϊκό νούμερο υποδημάτων
<b>cs</b>	Velikost - Velikost obuvi (Evropa)
<b>pl</b>	Rozmiar - Europejska numeracja obuwia
<b>lv</b>	Izmērs - Eiropas izmērs
<b>lt</b>	Dydis - Europos avalynės dydis
<b>et</b>	Suurus - Suurus Euroopa süsteemis
<b>sl</b>	Velikost - Velikost Evropa
<b>sk</b>	Veľkosť - Európska veľkosť topánok
<b>hu</b>	Méret - Európai cipőméret
<b>bg</b>	Размер - Европейски размер на обувката
<b>ro</b>	Mărime - Mărime europeană
<b>ru</b>	Размер - Европейский размер
<b>hr</b>	Veličina - Europski broj cipela
<b>zh</b>	尺码-鞋码 ( 欧盟 )
<b>ar</b>	المقاس - المقاس الأوروبي

<b>fr</b>	Stabilisation	Immobilisation
<b>en</b>	Stabilisation	Immobilisation
<b>de</b>	Stabilisierung	Ruhigstellung
<b>nl</b>	Stabilisatie	Immobilisatie
<b>it</b>	Stabilizzazione	Immobilizzazione
<b>es</b>	Estabilización	Inmovilización
<b>pt</b>	Estabilização	Imobilização
<b>da</b>	Stabilisering	Støtte
<b>fi</b>	Stabiloi	Immobilisoi
<b>sv</b>	Stabilisering	Immobilisering
<b>el</b>	Σταθεροποίηση	Akívntopoíñsn
<b>cs</b>	Stabilizace	Znehybnění
<b>pl</b>	Stabilizacja	Unieruchomienie
<b>lv</b>	Stabilizēšana	Imobilizācija
<b>it</b>	Stabilizavimas	Imobilizavimas
<b>et</b>	Stabiliseerimine	Liikumatuks muutmine
<b>sl</b>	Stabiliziranje	Imobilizacija
<b>sk</b>	Stabilizácia	Imobilizácia
<b>hu</b>	Stabilizálás	Rögzítés
<b>bg</b>	Стабилизация	Обездвижване
<b>ro</b>	Stabilizare	Imobilizare
<b>ru</b>	Стабилизация	Иммобилизация
<b>hr</b>	Stabilizacija	Imobilizacija
<b>zh</b>	稳定	固定
<b>ar</b>	الاستقرار	الثبات

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### en

## WALKING BOOT WITH RIGID SIDE UPRIGHTS AND INFLATABLE LINER

### Description/Destination

Walking boot with rigid side uprights to immobilize and stabilize the lower third of the leg, ankle and foot.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### Composition

Rigid components: polyamide - rubber - ethylene vinyl acetate - aluminium - thermoplastic polyurethane - polypropylene.

Textile components: polyamide - polyurethane - polyester.

### Properties/Mode of action

Immobilizes and stabilizes the lower third of the leg, ankle and foot.

90° rigid uprights on both sides.

Inflatable liner with integrated pump and valve.

Designed to ensure minimal dorsiflexion of the metatarsal bone. Curved, anti-slip sole that offers patients a natural, comfortable walking motion.

Product can be worn on either foot.

### Indications

Ankle/foot post-operative/post-traumatic immobilization.

Severe ankle/foot sprain.

Stable fracture of the lower third of the leg (tall version).

Conservative treatment of ankle/foot fracture.

Ankle/foot severe tendinopathy.

Conservative treatment of Achilles tendon rupture.

### Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of unstable fractures or fractures of the upper part of the tibia or fibula.

Do not use for patients weighing > 130 kg / 300lbs.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

### Properties

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

The choice of the device version is at the discretion of the healthcare professional.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

The systematic use of a long sock is recommended when wearing the device.

Ensure correct position and avoid folds when placing foam liner in the device.

Check the condition of the affected limb daily (with particular attention for patients with sensory deficit).

Do not bring the product into direct contact with greasy substances (ointment, cream, etc.).

Do not bring the product into direct contact with a dressing.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

Do not wear the device in a medical imaging machine.

Do not wear the device when driving a vehicle.

Do not expose the product to extreme temperatures (> 60°C).

It is recommended to adequately tighten the device to achieve immobilisation without restricting blood circulation. In the event

of pain, tingling or numbness, loosen the straps or deflate the air pads and ask a healthcare professional for advice.

It is recommended that the venous thromboembolic risk be assessed before any immobilisation is indicated. Follow the advice of a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

If the device comes into contact with water, dry the textile part and wipe the rigid part well with a dry cloth.

Prolonged immobilisation can sometimes cause muscle weakness.

When resuming walking, be careful of risks of falling.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

### Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Possible risk of back pain due to gait pattern modification.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### Instructions for use/Application

Before putting on the walking boot, make sure the tightening system has been fully unfastened and that the liner is fully deflated.

1. Take the walking boot out of its plastic cover, unfasten the self-fastening straps and open the liner.

2. Do not remove the protections from the side uprights. They must remain in place while you put the boot on.

3. Sit down and carefully place your lower leg and foot in the foam liner, checking that your heel is correctly positioned at the bottom of the boot  $\textcircled{A}$ . Close the foam liner, applying it firmly around the lower leg and foot  $\textcircled{B}$ .

4. Place your foot in the shell and your leg between the two side uprights.

5. For greater stability, the uprights can be adjusted.

6. Once you have checked that the side uprights are properly positioned, you can remove the protections  $\textcircled{C}$ .

The self-fastening surface on the inner part of the uprights attaches to the foam liner.

7. Fasten the walking boot again, tightening the self-fastening straps from bottom  $\textcircled{D}$  to top  $\textcircled{E}$ .

### Inflating/deflating the boot

For greater support, turn the valve as far as possible in the clockwise direction (inflate)  $\textcircled{F}$  then inflate the liner by squeezing the blue bulb  $\textcircled{G}$ . To deflate the liner, turn the valve as far as possible in the opposite direction (deflate)  $\textcircled{H}$  to release the air.

9. Try walking with the boot to check that it fits properly and that it is working correctly.

Taking off the walking boot: Loosen the straps and open the liner.

### Garmet care

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. If the device comes into contact with water, dry the textile part and wipe the rigid part well with a dry cloth.

### Rigid components:

Wash the rigid part with a moist cloth.

### Textile components:

The soft part can be fully removed for washing.

Replace in the original location before next use. Hand wash the liner in cold water. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron.

Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

### Storage

Store at room temperature, preferably in the original packaging.

### Disposal

Dispose of in accordance with local regulations.

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Thuasne UK Ltd