THUASNE


fr	Genouillère élastique	4
en	Elastic knee support	5
de	Elastische Kniebandage	7
nl	Elastische kniebrace	9
it	Ginocchiera elastica	11

	1	32 - 34 cm
	2	35 - 37 cm
	3	38 - 41 cm
	4	42 - 45 cm
	5	46 - 49 cm

fr	Circonférence de genou
en	Knee circumference
de	Knieumfang
nl	Omvang van de knie
it	Circonferenza del ginocchio

fr	Maintien	Compression	Effet proprioceptif
en	Support	Compression	Proprioceptive effect
de	Halt	Kompression	Propriozeptive Wirkung
nl	Fixatie	Compressie	Proprioceptieve werking
it	Sostegno	Compressione	Effetto proprioceettivo

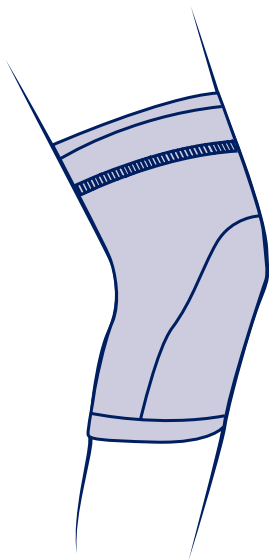MD

fr Dispositif médical

en Medical Device

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nl Medisch hulpmiddel

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en Single patient - multiple use

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ELASTIC KNEE SUPPORT

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. Bilateral product.

Available in 5 sizes.

The device is composed of an anatomically-shaped knitting integrating comfortable areas over the patella and at the back of the knee.

Composition

Cotton – polyamide – elastane.

Properties/Mode of action

Compressive elastic knit ensuring proprioception.

Indications

Knee pain and/or swelling management.

Proprioceptive support for rehabilitation.

Contraindications

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

History of venous or lymphatic disorders.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional:

- in case of doubt before use,
- in case of uncertain diagnosis.

Consult a healthcare professional and stop using the device:

- in case of discomfort, significant hindrance, pain, abnormal sensations,
- in the event of any modification in the product's performance.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

Slip the knee brace on the leg with your foot extended, making sure that the patella is properly positioned in the flexible knitted zone provided for this purpose.

Care

Machine washable at 40°C (delicate programme). If possible use a washing net. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

This medical device is a regulated health product with CE marking.
Keep this instruction leaflet.

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©Thuasne - 2024701 (2021-04)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186 –