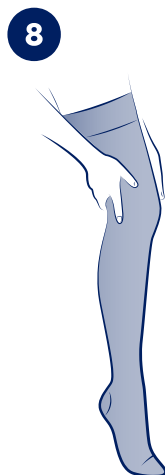
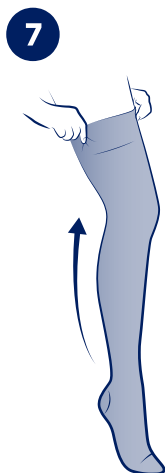
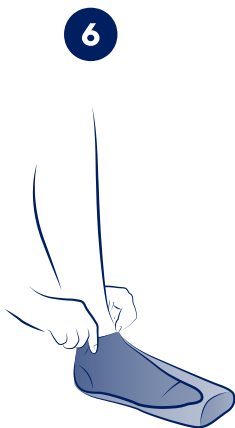
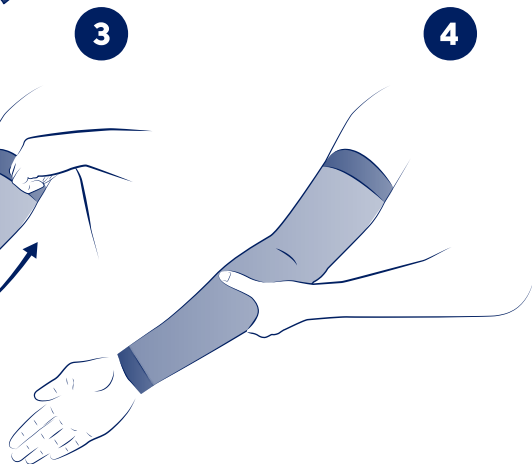
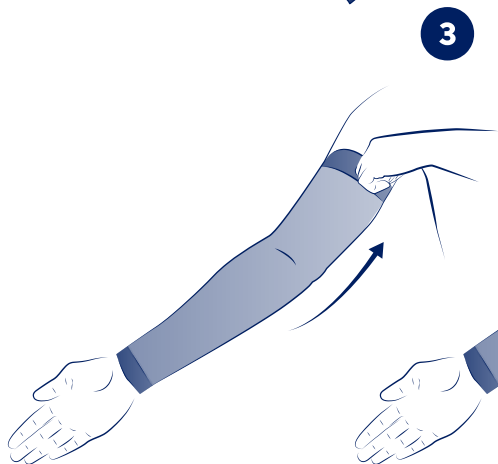
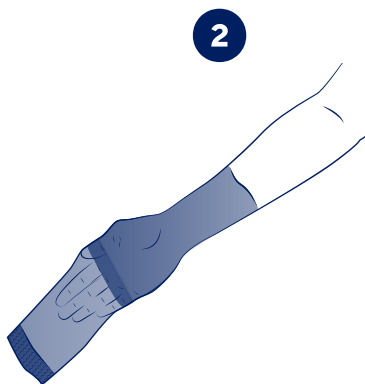
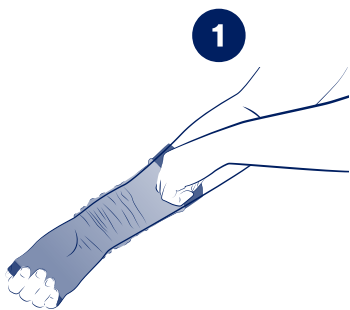
**THUASNE®**

## LYMPHATREX Essential

|    |                                                                          |    |
|----|--------------------------------------------------------------------------|----|
| fr | Dispositif de compression sur mesure patient .....                       | 3  |
| en | Made-to-measure patient compression device .....                         | 3  |
| de | Für Patienten maßgefertigtes Kompressionsprodukt .....                   | 4  |
| nl | Compressiehulpmiddel op maat van de patiënt .....                        | 5  |
| it | Dispositivo di compressione su misura paziente .....                     | 6  |
| es | Dispositivo de compresión a medida del paciente .....                    | 7  |
| pt | Dispositivo de compressão adaptado ao paciente .....                     | 8  |
| da | Kompressionsudstyr efter patientens mål .....                            | 9  |
| fi | Potilaan mittojen mukaan valmistettu kompressiotuote .....               | 10 |
| sv | Individanpassad kompressionsprodukt .....                                | 11 |
| el | Τεχνολογικό προϊόν συμπίεσης προσαρμοσμένο στον ασθενή .....             | 12 |
| cs | Kompresivní pomůcka na míru .....                                        | 13 |
| pl | Wyrób uciskowy na miarę .....                                            | 14 |
| lv | Pacientam pielāgots kompresijas izstrādājums .....                       | 14 |
| lt | Pagal paciento užsakymą pagaminta kompresinė priemonė .....              | 15 |
| et | Patsiendile eritellimusel valmistatud kohandatud kompressioontoode ..... | 16 |
| sl | Kompresijski pripomoček po meri pacienta .....                           | 17 |
| sk | Kompresná pomôcka vyrobená na mieru pre pacienta .....                   | 18 |
| hu | A páciens méretére készült kompressziós eszköz .....                     | 19 |
| bg | Компресивно изделие, адаптирано за пациента .....                        | 20 |
| ro | Dispozitiv de compresie realizat la comandă pentru pacient .....         | 21 |
| ru | Компрессионные изделия по меркам пациента .....                          | 22 |
| hr | Kompresijski proizvod po mjeri bolesnika .....                           | 23 |
| zh | 患者定制按压装置 .....                                                           | 24 |
| ja | カスタムメイドの圧迫装置 .....                                                       | 24 |
| ko | 환자 맞춤형 압박스타킹 .....                                                       | 25 |
| ar | تجهيز ضغط على مقياس المريض .....                                         | 27 |

fr

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Conservé cette notice.

en

# MADE-TO-MEASURE PATIENT COMPRESSION DEVICE

Description/Destination

Lymphatrex Essential is a range of compression devices made to the measures of the patient's limb by a circular knit.

Destination: The device is intended to be used directly on the skin of the lower or upper limbs, in adult or child patients suffering from at least one of the indications listed below.

This range is available in several models:

| Upper limb:                                                                                                                  | Lower limb:                                                                                        |
|------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------|
| <ul style="list-style-type: none"><li>• Armsleeve</li><li>• Armsleeve with mitten</li><li>• Mitten</li><li>• Glove</li></ul> | <ul style="list-style-type: none"><li>• Below-Knee</li><li>• Thigh-High</li><li>• Tights</li></ul> |

Various finish options are available, depending on the recommendations of your healthcare professional.

Composition

Lower limb: polyamide, elastane, polyester, polyethylene, cotton, polyurethane.  
Upper limb: polyamide, elastane, polyester, polyacetal, cotton, polyurethane.  
Anti-slip system: silicone, elastane, polyamide, polyester.

Properties/Mode of action

Lymphatrex Essential devices provide compression by applying pressure to the limb; they thus promote lymph circulation, help prevent edema formation and maintain its volume.

They are available in multiple AFNOR compression classes:

- Class 2 (15-20 mmHg)
- Class 3 (20-36 mmHg)

Indications

- Treatment of acute or chronic edema (for example: primary or secondary lymphedema of the upper or lower limbs).
- Class 2: additional treatment for the lymphedema maintenance phase.
- Class 3: during the maintenance phase, after initial reduction of the affected limb, to maintain the reduced volume.
- Symptomatic treatment of lipedema.

Contraindications

- Do not use in case of major dermatological conditions on an affected limb.
- Do not use in the event of known allergy to any of the components.
- Do not use in the event of phlegmasia cerulea dolens (painful blue edema with arterial compression) of the affected limb.
- Do not use in the event of septic thrombosis.

*Specific to the lower limbs:*

Do not use in the event of advanced diabetic microangiopathy (for compression > 30 mmHg).

Do not use in the event of decompensated heart failure.

Do not use in case of lower limb peripheral arterial disease (PAD) with ankle-brachial index (ABI) < 0.6.

Do not use in case of extra-anatomical bypass of the affected limb.

*Specific to the upper limbs:*

- Do not use in case of brachial plexus pathology.
- Do not use in case of vasculitis of the extremities.

Precautions

Strictly comply with your healthcare professional's prescription and recommendations for use.

The risk/benefit ratio and the adequate level of pressure must be regularly re-assessed by a healthcare professional in the event of:

- PAD with ABPI between 0.6 and 0.9
- Advanced peripheral neuropathy
- Weeping or eczematous dermatitis.

These compression devices are designed to be worn during the day.

In the event of discomfort, significant restriction, pain, variation in limb volume, deterioration of skin condition, infection, unusual sensations, change in the colour of the extremities, or changes in the performance of the product, remove the product and seek the advice of a healthcare professional.

Do not apply the product directly to an open wound without a dressing.

Due to the constant evolution of the pathology, regular follow-up with a healthcare professional is recommended to check that the compression device is still suitable for the pathology.

For some pathologies (or situations) such as venous thrombosis, the device must be used in combination with an anticoagulant treatment; consult a healthcare professional for advice.

In the event of lipedema, compression can be used to help relieve the symptoms. Seek advice from a healthcare professional for compression and other solutions for managing lipedema.

Do not use products on the skin (creams, ointments, oils, gels, patches...) before applying the device, it could damage the product.

Take care not to damage the device with your nails when fitting it.

Ensure that the anti-slip system can be applied to healthy undamaged skin.

Do not use when swimming.

The use of a medical device by a child, should be done under the supervision of an adult or a healthcare professional.

For hygiene, safety and performance reasons, do not reuse the product for another patient.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity or cutaneous dryness.

Any serious incident arising that is linked to the product should form the subject of a report to the manufacturers and the competent authority for the Member State in which the user and/or patient is domiciled.

Instructions for use/Application

Check the product integrity before every use.

Lymphatrex Essential compression devices should be worn directly on the skin. Seek advice from a healthcare professional.

Achieving the necessary pressure may require the overlaying of several compression devices.

*Putting on the armsleeve, armsleeve with mitten, armsleeve with glove:*

- Place your hand inside the garment ❶ and turn it back on itself to the hand opening. ❷
- Place your hand inside the garment, then carefully slide it over the limb to under the armpit (armsleeve, and armsleeve with mitten ), or to the wrist (mitten). ❸
- Avoid pulling the garment more than is necessary.
- Adjust the garment to ensure there are no wrinkles along the length of the limb.
- Once the garment is properly in place, massage the limb to ensure uniform compression. ❹

*Putting on below-knee, thigh-high and tights:*

- Place your hand inside the garment and turn it back on itself to the heel ❶
- Slip your foot into the garment until the heel is correctly positioned. ❷
- Carefully slide the garment over the limb to below the knee (below-knee), below the gluteal fold (thigh-high), or to the waist (tights). ❸
- Avoid pulling the garment more than is necessary.
- Adjust the garment to ensure there are no wrinkles along the length of the limb.
- Thigh-high or below-knee with anti-slip system: when putting on the garment, be careful not to grip the anti-slip part to avoid any damage.
- Once the garment is properly in place, massage the limb to ensure uniform compression. ❹

Care/Maintenance

- It is recommended that the garment be washed before it is used for the first time.
- Wash your garment every day, or after each use.
- Lymphatrex Essential compression devices are designed for daily use for periods of up to 6 months (i.e. 90 washes).
- They should then be replaced to maintain their technical characteristics and the effectiveness of the treatment.

*Machine washing:*

- Machine wash at 40°C.
- Use a washing product without softener. Do not use aggressive products (e.g., products containing chlorine).
- Place the garment in a wash bag to make sure other garments (particularly those with tough fasteners or zips) do not damage the knit of your orthosis
- Dry well away from any direct heat source (radiator, sun, etc.).
- Do not tumble dry.
- Do not iron.

Storage

Store the device at room temperature and away from direct sunlight.

Disposal

Dispose of in accordance with local regulations.

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Beine: Polyamid, Elasthan, Polyester, Polyethylen, Baumwolle, Polyurethan.

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©Thuasne - 2125203 (2024-02)
**THUASNE UK Ltd**
Thuasne SAS - SIREN/RCS Nanterre 542 091 186