THUASNE®


Ortel C4 Vario

de	Gebrauchsanweisung - Cervical-Orthese
en	Instructions for use - Cervical orthosis
fr	Consignes d'utilisation - Collier cervical
it	Istruzioni per l'uso - Ortesi cervicali
es	Instrucciones de utilización - Órtesis cervical
pt	Instruções de uso - Órtese cervical
nl	Gebruiksaanwijzing - Cervicaal orthese
da	Brugsanvisning - Cervikal-ortese
sv	Bruksanvisning - Cervikal-ortos
fi	Käyttöohje - Kaularankatukiside
el	Οδηγίες χρήσης - Αυχενικός κηδεμόνας
cs	Návod k použití - Pevný krční límec
pl	Sposób użytkowania - Orteza szyjna
et	Kasutusjuhend - Tservikaalne ortoos
lv	Lietošanas instrukcija - Cervikālā ortoze
lt	Naudojimo instrukcija - Ortopedinis kaklo įtvaras
sk	Návod na použitie - Pevný krčný golier
sl	Navodila za uporabo - Vratna ortoza
hu	Használati utasítás - Nyaki ortézis
ro	Instrucțiuni de utilizare - Orteză cervicală
bg	Инструкция за употреба - Цервикална ортеза
hr	Upute za uporabu - Cervikalna ortoza


(2)


(3)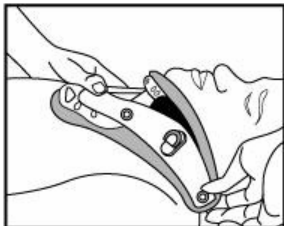

(4)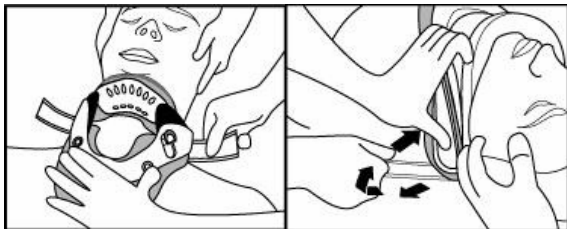

(5)

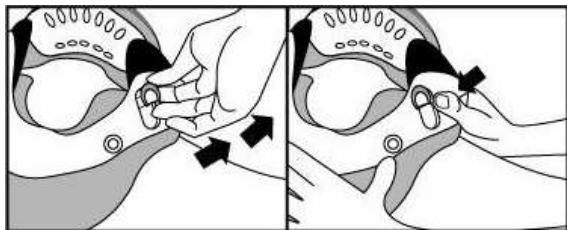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en

Purpose

Ortel C4 Vario is an orthosis for stabilising the cervical spine. The orthosis may only be used for cervical spine treatment.

Indications

Post-operative care, severe spondylarthritis, severe cervical syndrome, physical wear and tear, slipped cervical discs

Contraindications

Skin lesions or diseases in the affected area.

Side effects

None reported when applied and worn as intended.

Fitting instructions

The Ortel C4 Vario must be fitted individually to each patient by qualified experts. At least two people are needed for the first fitting: One to hold the head and neck of the patient in the correct position while the other fits the orthosis.

- (1) Place the patient in a neutral position with the head aligned to the centre.
- (2) Place the neck part of the orthosis at the centre back of the throat.
- (3) Then place the front of the Ortel C4 Vario at the centre of the back part.
- (4) Hold the front of the orthosis firmly and place the ends tightly around the patient's neck. Fasten the orthosis by alternately tightening the rip tape bands, making sure they are the same length on both sides. If necessary, the surplus at the ends can be cut off with scissors. If the orthosis has been fitted correctly, it should no longer be possible for the chin to slip.
- (5) The option of adjusting the angle of the chin ensures optimal pressure distribution and an improved fit to the anatomical conditions. Pull out the buttons for angle adjustment to fine tune the correct chin position. Adjust the chin position and fasten in

place by pushing the buttons back in. Please ensure that the same height adjustment is performed on both sides.

Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Care instructions

- Before washing, take the pads out of the orthosis.
- Soak the pads in lukewarm soapy water.
- Squeeze the water out of the pads and dry away from any heat sources.
- All plastic parts can be cleaned with a damp cloth.

Material composition

32% Polycarbonate, 30% Polypropylene, 10% Polyurethane, 10% Polyester, 10% Cotton, 8% Polyamide

Please keep these instructions in a safe place

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Thuasne Cervitex Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
Thuasne Scandinavia AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
Thuasne Deutschland GmbH