
THUASNE®

GenuPro Comfort

fr	Genouillère élastique rotulienne.....	5
en	Elastic patellar knee support.....	6
de	Elastische Patella-Kniebandage	8
nl	Elastische kniebrace voor de knieschijf.....	10
it	Ginocchiera elastica rotulea.....	11
es	Rodillera elástica rotuliana	13
pt	Joelheira elástica rotuliana.....	15
da	Elastisk knæbind til knæskallen.....	16
fi	Joustava polvilumpiotuki.....	18
sv	Elastiskt knäskålstöd	19
el	Ελαστική επιγονατίδα για χρόνιες παθήσεις.....	21
cs	Elastická česková ortéza	23
pl	Elastyczna orteza rzepkowa stawu kolanowego.....	24
lv	Elastīga ceļgala ortoze	26
lt	Tamprus girnelės įtvaras.....	28
et	Elastne põlvvetugi põlvkedra toestusega	29
sl	Elastična opornica za pogačico.....	31
sk	Elastická patelárna kolenná ortéza	32
hu	Rugalmas patella térdrögzítő.....	34
bg	Еластична пателарна скоба за коляното.....	36
ro	Genunchieră elastică rotuliană.....	38
ru	Эластичный наколенник для фиксации коленного сустава.....	39
hr	Elastična ortoza za čašicu koljena	41
zh	彈力髌骨护膝.....	43
ar	دعامة ركبة رضفية مرنة.....	45


Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

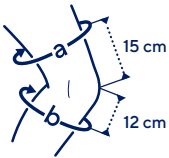		(a)	(b)
	0	35 - 37 cm	29,5 - 32 cm
	1	37 - 41 cm	32 - 34,5 cm
	2	41 - 45 cm	34,5 - 37 cm
	3	45 - 50 cm	37 - 39,5 cm
	4	50 - 55 cm	39,5 - 42 cm
	5	55 - 61 cm	42 - 44,5 cm
	6	61 - 67 cm	44,5 - 47 cm

fr	(a) Circonférence de la cuisse (15 cm au-dessus du genou)	(b) Circonférence du mollet (12 cm au-dessous du genou)
en	(a) Thigh circumference (15 cm above the knee)	(b) Calf circumference (12 cm below the knee)
de	(a) Oberschenkelumfang (15 cm über dem Knie)	(b) Wadenumfang (12 cm unter dem Knie)
nl	(a) Omvang van de dij (15 cm boven de knie)	(b) Omvang van de kuit (12 cm onder de knie)
it	(a) Circonferenza della coscia (15 cm al di sopra del ginocchio)	(b) Circonferenza del polpaccio (12 cm al di sotto del ginocchio)
es	(a) Circunferencia del muslo (15 cm por encima de la rodilla)	(b) Circunferencia de la pantorrilla (12 cm por debajo de la rodilla)
pt	(a) Circunferência da coxa (15 cm acima do joelho)	(b) Circunferência da barriga da perna - (12 cm abaixo do joelho)
da	(a) Lårets omkreds (15 cm over knæet)	(b) Læggets omkreds (12 cm under knæet)
fi	(a) Reiden ympärys (15 cm polven yläpuolelta)	(b) Pohkeen ympärys (12 cm polven alapuolelta)
sv	(a) Omkrets runt låret (15 cm ovanför knäet)	(b) Omkrets runt vad (12 cm under knäet)
el	(a) Περίμετρος του μηρού (15 εκ. πάνω από το γόνατο)	(b) Περίμετρος της γάμπας (12 εκ. κάτω από το γόνατο)
cs	(a) Obvod stehna (15 cm nad kolenem)	(b) Obvod lýtky (12 cm pod kolenem)
pl	(a) Obwód uda (15 cm powyżej kolana)	(b) Obwód łydki (12 cm poniżej kolana)
lv	(a) Augšstilba apkārtmērs (15 cm virs ceļgala)	(b) Apakšstilba apkārtmērs (12 cm zem ceļgala)
lt	(a) Šlaunies apimtis (15 cm virš kelio)	(b) Blauzdos apimtis (12 cm žemiau kelio)
et	(a) Reie ümbermõõt (15 cm põlvest kõrgemal)	(b) Sääremarja ümbermõõt (12 cm põlvest allpool)
sl	(a) Obseg stegna (15 cm nad kolenom)	(b) Obseg goleni (12 cm pod kolenom)
sk	(a) Obvod stehna (15 cm nad kolenom)	(b) Obvod lýtky (12 cm pod kolenom)
hu	(a) Comb körfogata (a térd felett 15 cm-rel)	(b) Vádli körfogata (a térd alatt 12 cm-rel)
bg	(a) Обиколка на бедрото (15 cm над коляното)	(b) Обиколка на прасеца (12 cm под коляното)
ro	(a) Circumferința coapsei (la 15 cm deasupra genunchiului)	(b) Circumferința gambei (la 12 cm sub genunchi)
ru	(a) диаметр окружности бедра (на 15 см выше колена)	(b) диаметр окружности голени (на 12 см ниже колена)
hr	(a) Opseg bedra (15 cm iznad koljena)	(b) Opseg lista (12 cm ispod koljena)
zh	(a) 大腿周长 (膝盖上方15厘米处)	(b) 小腿周长 (膝盖下方12厘米处)
ar	(a) محيط الفخذ (15 سم فوق الركبة)	(b) محيط رولة الساق (12 سم تحت الركبة)

fr	Pression localisée	Compression	Effet proprioceptif
en	Targeted pressure	Compression	Proprioceptive effect
de	Lokaler Druck	Kompression	Propriozeptive Wirkung
nl	Lokale druk	Compressie	Proprioceptieve werking
it	Pressione localizzata	Compressione	Effetto propriocettivo
es	Presión localizada	Compresión	Efecto propioceptivo
pt	Pressão localizada	Compressão	Efeito propioceptivo
da	Lokalt tryk	Kompression	Proprioceptisk effekt
fi	Paikallinen paine	Kompressio	Liikeaistiin kohdistuva vaikutus
sv	Lokaliserat tryck	Kompression	Proprioceptiv effekt
el	Εντοπισμένη πίεση	Συμπίεση	Αποτέλεσμα ιδιοδεκτικότητας
cs	Lokalizovaný tlak	Komprese	Proprioceptivní účinek
pl	Ucisk miejscowy	Kompresja	Propriocepcja
lv	Lokalizēts spiediens	Kompresija	Proprioceptīvs efekts
lt	Konkrečios vietos suspaudimas	Kompresija	Propriocepcinis poveikis
et	Paikne surve	Kompressioon	Proprioceptiivne toime
sl	Lokaliziran pritisk	Kompresija	Proprioceptivni učinek
sk	Presne zacielený tlak	Kompresia	Proprioceptívny účinok
hu	Lokalizált nyomás	Szorítás	Proprioceptív hatás
bg	Локализирано налягане	Компресия	Проприоцептивен ефект
ro	Presiune localizată	Compresie	Efect proprioceptiv
ru	Локализованное давление	Компрессия	Проприоцептивный эффект
hr	Lokalizirani pritisak	Kompresija	Učinak proprioceptivnog vježbanja
zh	局部施压	压紧	本体感
ar	مفعول استقبال الحس العميق الضغط موضعي		

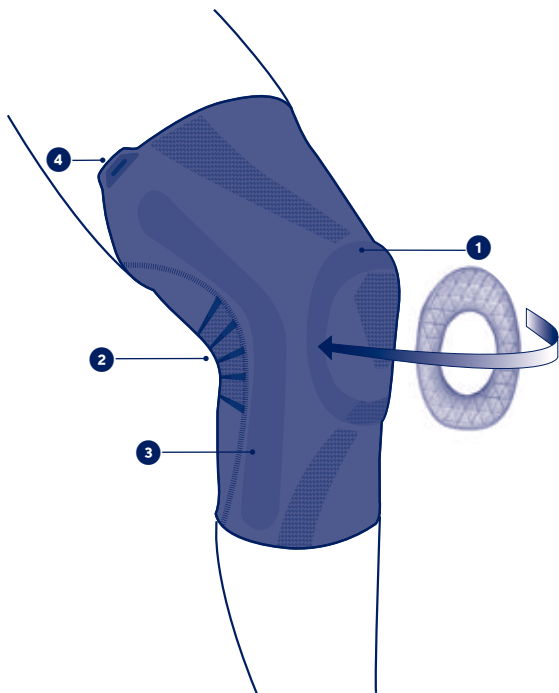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ELASTIC PATELLAR KNEE SUPPORT

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Bilateral product. Available in 7 sizes.

The device is composed of:

- a removable patellar insert ❶,
- an anatomically-shaped knitting integrating a soft knit at the back part of the knee brace ❷,
- flexible lateral stays ❸.

Composition

Textile components: polyamide - polyester - polyurethane - cotton - elastane - viscose.

Rigid components: steel - polyethylene - thermoplastic elastomer - silicone - polyurethane glue.

Properties/Mode of action

Conformable insert to support patella.

Elastic fabric with standardized compression (23-32 mmHg/ 31-43 hPa) ensuring good support and proprioception.

Specific construction of the knitting facilitating the fitting: back part of the knee brace ❷, pull tabs ❹.

Viscose thread presence for more softness.

The knee support is maintained on the leg thanks to flexible side stays and anti-slip system.

Indications

Anterior knee or patella pain.

Mild patellar instability.

Patella tendinopathy (including jumper's knee syndrome).

Knee pain and/or swelling management.

Knee osteoarthritis (mild to moderate).

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

If the measurements taken do not correspond to the same size, favor the size corresponding to measurement (a).

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

This product is intended for the treatment of a given condition. Its duration of use is limited to this treatment only.

For hygiene and performance reasons, do not re-use the product for another patient. In the event of history of venous or lymphatic disorders, consult a healthcare professional.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

The use of a medical device by a child should be done under the supervision of an adult or a healthcare professional.

Do not wear the device in a medical imaging machine.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

- Have the patient sit on the edge of a chair, and have the patient bend his/her knee to a position of approximately 30-60° of flexion.
To make it easier to put on, use the pull tabs at the top of the knee brace ❹.
- Slip the knee support on the leg with your foot extended.
- Make sure that your knee cap is positioned in the centre of the patella insert ❶.
- Check that the knee support is pulled completely taut over the leg.

Care/Maintenance

Machine washable at 30°C (delicate programme). If possible use a washing net. It is recommended to remove the patellar insert ❶ before washing. Take note of its original position (flat area outside, wide area above), in order to reposition it correctly. Do not machine wash the device more than 20 times. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2029502 (2024-07)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186