
**THUASNE®**

# Epi PROMASTER

- de Gebrauchsanweisung - Ellenbogengelenkbandage mit verstärktem propriozeptiven Effekt
- en Instructions for use - Anti-epicondylitis bandage with reinforced proprioceptive effect
- fr Consignes d'utilisation - Coudière anti-épicondylite à effet proprioceptif renforcé
- it Istruzioni per l'uso - Gomitiera con effetto propriocettivo rinforzato
- es Instrucciones de utilización - Codera con efecto propioceptivo reforzado
- pt Instruções de uso - Órtese para cotovelo com efeito propioceptivo reforçado
- nl Gebruiksaanwijzing - Verband voor het ellebooggewricht met versterkt proprioceptief effect
- da Brugsanvisning - Albuebind med forstærket proprioceptiv virkning
- sv Bruksanvisning - Armbågsledsforband med förstärkt proprioceptiv verkan
- fi Käyttöohje - Kyynärside vahvistetulla proprioseptiivisellä vaikutuksella
- el Οδηγίες χρήσης - Επιπαικωνίδα με ενισχυμένη ιδιοδεκτικότητα
- cs Návod k použití - Bandáž lokte se zesíleným proprioceptivním efektem
- pl Sposób użytkowania - Orteza stawu łokciowego ze wzmocnionym efektem proprioceptywnym
- et Kasutusjuhend - Küünarliigese bandaaž tugevdatud propriotseptiivse efektiga
- lv Lietošanas instrukcija - Elkoņa locītavas bandāža ar uzlabotu proprioceptīvo efektu
- It Naudojimo instrukcija - Proprioceptinio poveikio sutvirtintas alkūnės įtvaras
- sk Návod na použitie - Ortéza lakťového kĺbu so zosilneným proprioceptívnym účinkom
- sl Navodila za uporabo - Opora za komolec z izboljšanim proprioceptivnim učinkom
- hu Használati utasítás - Könyökízület-rögzítő megerősített proprioceptív hatással
- ro Instrucțiuni de utilizare - Bandaj pentru încheietura cotului cu efect proprioceptiv pronunțat
- bg Инструкция за употреба - Ортеза за лакътна става с подсилено proprioцептивно действие
- hr Upute za uporabu - Bandaža za zglobov lakta s pojačanim proprioceptivnim učinkom

| Größe <sup>1</sup> | Unterarmumfang <sup>2</sup> |             |
|--------------------|-----------------------------|-------------|
|                    | cm                          | inch        |
| XS                 | 21 - 23                     | 8 ¼ - 9     |
| S                  | 23 - 25                     | 9 - 9 ¾     |
| M                  | 25 - 27                     | 9 ¾ - 10 ½  |
| L                  | 27 - 29                     | 10 ½ - 11 ½ |
| XL                 | 29 - 31                     | 11 ½ - 12 ¼ |
| XXL                | 31 - 34                     | 12 ¼ - 13 ½ |

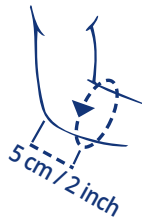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**Anti-epicondylitis bandage with reinforced proprioceptive effect****Purpose:**

Epi PROMASTER is a compression bandage with pads for treating elbow irritation. The bandage may only be used for elbow treatment.

**Properties:**

- Brace made of elastic knit
- 3D knit & pad opening at the condyles
- Epicondyle pads
- Latex-free

**Indications:**

Epicondylitis; Post-traumatic, postoperative and chronic inflammation; Sprains; Joint effusions and oedema linked to arthritis and joint disease

**Contraindication:**

Do not apply the product in direct contact with broken skin.

**Side effects:**

No side effects are as yet known for correct use.

**Fitting instructions:**

Make sure that the epicondyles are placed correctly in the special knit area. The brace should be worn directly against the skin.

**Precautions:**

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

**Washing instructions:**

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

**Composition:**

78 % polyamide, 14 % elastane, 5 % cotton, 2 % polyurethane, 1 % polyester

**Explanation of the reinforced proprioceptive effect:**

Orthoses in the PROMASTER range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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Thuasne Scandinavia AB · Box 95 · 131 07 Nacka
**Thuasne UK Ltd**
**Thuasne Deutschland GmbH**