
THUASNE®


Dynacross

fr	Ceinture de soutien lombaire	5
en	Lumbar support belt	6
de	LWS-Stabilisierungsorthese	7
nl	Lumbale rugbrace	8
it	Cintura di sostegno lombare	9
es	Faja de sujeción lumbar	10
pt	Cinta de suporte lombar	11
da	Lændestøttebælte	12
fi	Lannerangan tukivyö	13
sv	Ryggstöd för ryggen	14
el	Ζώνη οσφυϊκής υποστήριξης	15
cs	Bederní podpurný pás	17
pl	Podtrzymujący pas lędźwiowy	18
lv	Muguras jostas daļas atbalsta josta	19
lt	Palaikomasis juosmens diržas	20
et	Nimmepiirkonna tugikorsett	21
sl	Oporni križni pas	22
sk	Podporný drevkový pás	23
hu	Ágyéki fűző	24
bg	Лумбален колан	25
ro	Centură de susținere lombară	26
ru	Опорный поясничный корсет	27
hr	Lumbalni potporni pojas	29
zh	腰部稳定护带	30
ar	حزام لدعم أسفل الظهر	31


Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegółowy użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiunile de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال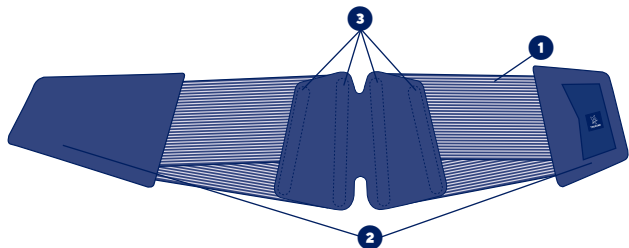



(A)

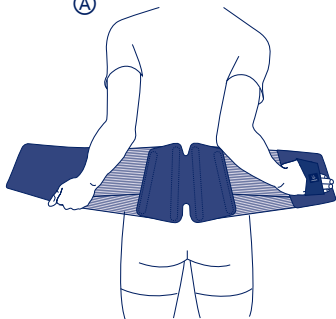
(B)

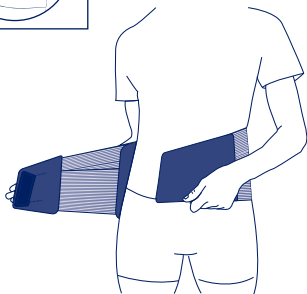(C)

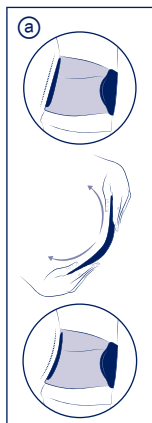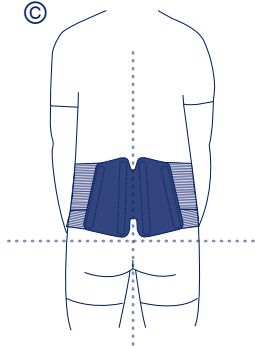	1	60 - 80 cm
	2	75 - 95 cm
	3	90 - 110 cm
	4	105 - 125 cm
	5	120 - 140 cm

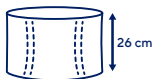fr	Tour de taille	Hauteur : 26 cm
en	Waist circumference	Height: 26 cm
de	Taillenumfang	Höhe: 26 cm
nl	Tailleomvang	Hoogte: 26 cm
it	Circonferenza vita	Altezza: 26 cm
es	Contorno de cintura	Altura: 26 cm
pt	Perímetro de cintura	Altura: 26 cm
da	Taljemål	Højde: 26 cm
fi	Vyötärön ympärys	Korkeus: 26 cm
sv	Midjemått	Höjd: 26 cm
el	Περίμετρος μέσης	Ύψος: 26 εκ.
cs	Obvod pasu	Výška: 26 cm
pl	Obwód w talii	Wysokość: 26 cm
lv	Vidukļa apkārtmērs	Augstums: 26 cm
lt	Juosmens apimtis	Aukštis: 26 cm
et	Vööümbermõõt	Kõrgus: 26 cm
sl	Obseg pasu	Višina: 26 cm
sk	Obvod pásu	Výška: 26 cm
hu	Derék bősége	Magasság: 26 cm
bg	Обиколка на талията	Височина: 26 cm
ro	Circumferința taliei	Înălțime: 26 cm
ru	Обхват талии	Высота: 26 см
hr	Opseg struka	Visina: 26 cm
zh	腰围	高度 : 26 cm
ar	محيط الخصر	الارتفاع: 26 سم

fr	Compression	Maintien	Éducation posturale
en	Compression	Support	Postural education
de	Kompression	Halt	Bessere Körperhaltung lernen
nl	Compressie	Fixatie	Posturale training
it	Compressione	Sostegno	Educazione posturale
es	Compresión	Sujeción	Educación postural
pt	Compressão	Manutenção	Educação postural
da	Kompression	Støtte	Postural genoptræning
fi	Kompressio	Tuki	Asentokoulutus
sv	Kompression	Stöd	Postural träning
el	Συμπίεση	Συγκράτηση	Σωστή στάση σώματος
cs	Komprese	Stabilizace	Správné držení těla
pl	Kompresja	Podtrzymywanie	Edukacja dotycząca postawy
lv	Kompresija	Atbalsts	Stājas korekcija
lt	Kompresija	Palaikymas	Laikysenos taisymas
et	Kompressioon	Hooldus	Rühitreening
sl	Kompresija	Oprijem	Učenje telesne drže
sk	Kompresia	Podpora	Cvičenie posturálnych svalov
hu	Szorítás	Támasz	Tartásjavító oktatás
bg	Компресия	Поддръжка	приучаване към правилна стойка
ro	Compresie	Susținere	Educație posturală
ru	Компрессия	Стабилизация	Исправление осанки
hr	Kompresija	Potpورا	Učenje položaja
zh	压紧	稳定	姿势练习
ar	الضغط	الدعم	تربية القوام

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Description/Destination

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LUMBAR SUPPORT BELT

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. This device can be used to support rehabilitation, particularly when resuming activities.

Elastic fabric. ①

Abdominal panels with hook-and-loop surface and finger-loops. ②

Conformable dorsal stays. ③

Composition

Textile components:

Polyamide - polyester - elastane.

Non-textile components:

Brushed tempered steel - polyoxymethylene - polyethylene.

Properties/Mode of action

The device relieves lower back pain by supporting the spine through an increase in intra-abdominal pressure. This reduces the load on the intervertebral discs.

Indications

Non-specific low back pain (acute, subacute and chronic phases).

Spinal stenosis.

Spine stability.

Contraindications

Do not use in the event of known allergy to any of the components.

Do not use on pregnant women.

Do not use in the event of a hiatus hernia.

Do not use in the event of bone cancer with spinal metastases.

Do not use in the event of circulatory, pulmonary or cardiovascular problems in patients for whom an increase in arterial blood pressure is not recommended.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Wear the device above thin clothing.

Strictly comply with your healthcare professional's prescription and recommendations for use.

This product is intended for the treatment of a given condition. Its duration of use is limited to this treatment only.

Do not wear the device while sleeping.

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

The use of a medical device by a child should be done under the supervision of an adult or a healthcare professional.

Measure the waist circumference of the patient at the mid-distance between the iliac crests and floating ribs.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not wear the device in a medical imaging machine.

Undesirable side-effect

This device can cause skin reactions (redness, itching, irritations...).

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended that a healthcare professional adjust the stays to the shape of the patient's back before the belt is worn for the first time. ④ Position the belt around the waist, using abdominal panel and the finger loop located on the opposite abdominal panel. ⑤

The Thuasne® logo indicates which way up the belt should be put on.

Pull the finger loop and the opposite abdominal panel simultaneously and evenly.

Fasten the belt, right side over left side, adjusting to the desired compression. ⑥

The belt must be centred on the spine: the dorsal stays should be positioned on either side of the spine. ⑦

The bottom of the belt must stop at the top of the crease between the buttocks. ⑧

The level of compression can be adjusted as required throughout the day.

Garment care

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Close the self-fastening tabs before washing. Machine washable at 30°C (delicate programme). It is not recommended to machine wash the device more than 15 times (average: one wash per month). Do not dry clean. Do not use detergents, fabric softeners or aggressive

products (products containing chlorine, etc.). Do not tumble-dry. Do not iron. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations. Keep this instruction leaflet.

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Die Orthese auf der rechten Seite über der linken Seite schließen, dabei den gewünschten Spannungsgrad einstellen. ©
THUASNE UK Ltd**
Thuasne SAS – SIREN/RCS Nanterre 542 091 186