THUASNE

Malleo Dynastab Vario

D: Sprunggelenkorthese zur Mobilisierung in definierter Position, abrüstbar
GB/USA: Ankle brace for mobilising in a defined position, modular concept
F: Cheville de mobilisation dans une position définie, pouvant être réduite
I: Ortesi per l'articolazione tibio-tarsica per la mobilizzazione in una posizione definita, scomponibile in più elementi
E: Órtesis evolutiva de tobillo para la movilización en una posición definida
P: Ortótese de tornozelo para mobilização numa posição definida, adaptável
NL: Enkelorthese voor de mobilisatie in een gedefinieerde positie, aanpasbaar
DK: Ankelortese til mobilisering i defineret position, trinvist aftagelig
S: Fotledsortos för rörlighet i definierad position, kan tas bort
FIN: Nilkkatukiside, joka mahdollistaa koukistus- ja ojennusliikkeet, säädettävä
GR: Νάρθηκας ποδοκνημικής για κινητοποίηση σε καθορισμένη θέση, ανοιγόμενος
CZ: Ortéza hlezenního kloubu pro mobilizaci v definované poloze, modulární
PL: Orteza stawu skokowego do aktywizacji w określonej pozycji, z możliwością redukcji
EST: Hüppeliigeseortoos kindlasse asendisse mobiliseerimiseks, eemaldatav
LV: Potītes ortoze mobilizēšanai definētā pozīcijā, ar noņemamu aprīkojumu
LT: Čiurnos sąnario mobilizavimo tam tikroje padėtyje įtvirtas, nuimamas
SK: Ortéza členkového kĺbu pre mobilizáciu vo vymedzenej polohe, prispôsobiteľná
SLO: Orteza za skočni sklep za mobilizacijo na določenem položaju, prilagodljiva
H: Bokaízület-ortézis meghatározott pozícióban történő mobilizáláshoz, moduláris felépítésű
RO: Orteză pentru gleznă pentru mobilizare în poziție definită, demontabilă
BG: Ортеза за мобилизация на глезенната става в определена позиция, сменяща се
HR: Ortoza gležnja za mobilizaciju u određenom položaju, rastavljiva

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Ankle brace for mobilising in a defined position, modular concept

Purpose

Malleo Dynastab Vario is an ankle brace with added stabilising element and strap for early functional mobilisation. The orthosis may only be used for foot treatment.

Properties

- Rear entry for pain-free application of the orthosis
- Wide-area compression reduces swelling and haematoma
- Stabilising element and strap for gradual mobilisation
- Modern pull-and-fasten system makes the brace easy to put on and close
- Two additional stabilising elements

Indications

- Acute ankle injuries/grade II and III sprains
- Acute and chronic instability of the upper and lower ankle joints
- Post-operative care following ligament surgery on the ankle
- Acute inflammatory ankle disorders

Contraindications

- Open wounds or skin disorders in the area of application
- Circulation problems
- Lymph drainage problems

Side effects

To date, no side effects have been registered during proper use.

Precautionary measures

Follow the instructions given by your doctor or orthopaedic technician. If you experience severe pain or discomfort, please consult your doctor or orthopaedic technician. Store at room temperature, preferably in the original packaging. To guarantee its effectiveness and for reasons of hygiene, the product may not be passed on to other patients. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

How it works/Therapy

The orthosis combines firm base material with a rigid plastic element and crossover strap to

immobilise and stabilise the upper and lower ankle joints. Pronation, supination and plantar flexion are restricted; talus prolapse is counteracted. Immobilisation and stabilisation relieve the strain on injured structures, protect them, and prevent detrimental joint movements. The modular design is variable to match the different stages of the therapy process, i.e. extended mobility is enabled in a controlled process by removing the plastic element and eventually the strap; the therapy can be adapted to the appropriate phases of healing.

Phase 1 (orthosis + plastic element + crossover strap):

During the acute phase, immediately after the trauma, the foot must be immobilised and stabilised. The full brace, i.e. complete with stabilising element and strap, offers maximum support. The foot is supported, swelling reduction is promoted, and pain is reduced.

Phase 2 (orthosis + crossover strap):

To further promote the reduction of swelling and to improve muscular stabilisation, the level of immobilisation can be reduced in the second phase, and the foot gradually mobilised. By dispensing with the U-shaped plastic element, controlled movement is now possible. Together with the crossover strap, the base element of the orthosis continues to guarantee the necessary stability.

Stability can be enhanced even further by inserting the two separate reinforcements in the side pockets, once the U-shaped plastic element has been removed. For correct positioning, the reinforcements must be pushed in the direction of the arrow (punched-out arrow in grey-fabric cover) upwards into the pocket (Fig. 7). The arrow in the fabric cover of the reinforcement must remain visible. Turn the lower part of the pocket over the lower edge of the reinforcements once they are pushed in.

We recommend continuing to wear the U-shaped plastic element at night during this phase to protect structures that have had time to heal during the day and to prevent them from moving uncontrollably and suffering renewed injury.

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To further enhance mobilisation, the crossover strap is also removed in the last phase to enable the structures to become increasingly accustomed to the load and to strengthen them in this third phase of the therapy.

Stability can be enhanced even further by inserting the two separate reinforcements in the side pockets. For correct positioning, the reinforcements must be pushed in the direction of the arrow (punched-out arrow in grey-fabric cover) upwards into the pocket (Fig. 7). The arrow in the fabric cover of the reinforcement must remain visible. Turn the lower part of the pocket over the lower edge of the reinforcements once they are pushed in.

Putting on the orthosis

Insert the side bars of the U-shaped plastic element into the two pockets on the outside of the orthosis (Fig. 1) and push them all the way into the pockets. Both of the pockets feature an opening for this purpose, located more or less at ankle bone height. During insertion, make sure the tip of the plastic element is pointing towards the front of the foot.

Next, attach the crossover strap to the orthosis by positioning the centre of the strap beneath the front part of the orthosis. The rear edge of the strap should rest against the anti-slip fixture on the U-shaped plastic element. The triangle in the middle of the crossover strap is used to fix the strap to the orthosis; it fits exactly into the recess on the U-shaped plastic element, i.e., the tip of the triangle must point towards the front of the foot (Fig. 2).

To put the brace on, open both of the straps at the back (Fig. 3) and place the foot in the orthosis from the rear (Fig. 4).

To tighten the traction system, pull briefly on the straps before fastening them (Fig. 5). When pulling the straps tight, make sure the two side parts do not move (if the two side parts move too far forwards or backwards, please re-align them to ensure an optimum fit), and that the foot is at an angle of 90°. Then fasten the straps by threading the relevant ends through the eyes on the opposite sides and then pulling the strap again to tighten the lacing system. Fix the strap in place by turning the end over and fastening it to the strap. Close the lower strap first, then the upper strap.

Then close the crossover strap by taking both ends diagonally across the instep to the sides at the rear (Fig. 6) and fastening them both to the orthosis. The two parts of the strap should cross over at the front, at ankle height.

When closing the orthosis and the strap, make

sure that the orthosis fits tightly, but without constricting.

Taking off the orthosis

To take the orthosis off, loosen the strap across the instep and the two straps at the back. The brace can then be pulled off the foot towards the front.

Care

- First, remove the plastic element (can be wiped clean with a damp cloth)
- Close all the Velcro fastenings prior to washing
- Do not use any cleaning agents, conditioners or aggressive products (containing chlorine)
- Squeeze out the water thoroughly
- Dry away from any heat sources

Material composition

Orthosis: 53% polyamide, 14% chloroprene, 13% polyurethane, 10% polyethylene, 9% polyester, 1% elastane

Strap: 78% polyamide, 22% polyurethane

Plastic splint: 92% polypropylene, 8% synthetic rubber

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Thuasne Deutschland GmbH