THUASNE



fr	Poignet de maintien	4
en	Wrist support	5
de	Handgelenksstütze	6
nl	Polsbrace	8
it	Polsino di sostegno	9

	1	13 - 16 cm
	2	16,5 - 19,5 cm
	3	20 - 23 cm

fr	Tour de poignet
en	Wrist circumference
de	Handgelenkumfang
nl	Polsomtrek
it	Circonferenza del polso

Immobilisation
Immobilisation
Ruhigstellung
Immobilisatie
Immobilizzazione

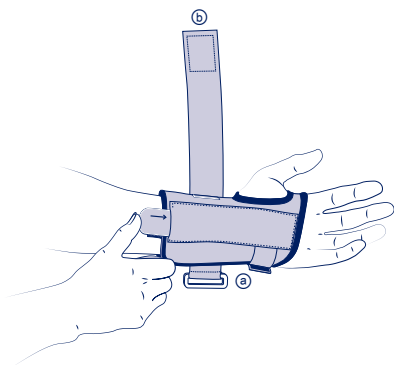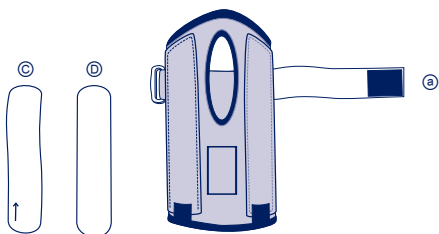MD

fr Dispositif médical

en Medical Device

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nl Medisch hulpmiddel

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en Single patient - multiple use

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WRIST SUPPORT

Description/Destination

This device is a wrist splint providing joint immobilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Polyamide - polyester - cotton - elastane - polyoxymethylene - polyurethane - aluminium.

Properties/Mode of action

Immobilisation of the wrist via a rigid, malleable and anatomical palmar stay and a dorsal stay.

Aerated, foam lined fabric.

Removable strap to adjust compression level.

Bilateral product.

Indications

Wrist sprain.

Wrist tendinopathies.

Rheumatoid arthritis.

Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional:

- in case of doubt before use,
- in case of uncertain diagnosis

Consult a healthcare professional and stop using the device:

- in case of discomfort, significant hindrance, pain, abnormal sensations.
- in case of changes in the device's performance.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated. Pull the orthosis over the wrist to be immobilised, leaving the straps on the outside, and find the two stays covers.

Slip the metal stay ③ into the stay cover located on the inside of the hand (from the bottom).

To do this, the inscription on the stay must be towards the elbow and facing you. Once on, it must mould to the shape of your palm.

Insert the plastic stay ④ into the second stay cover.

Strap ⑤: wrap around the hand and insert the strap into the tightening buckle.

Adjust the strap using the self-fastening tab.

Strap ⑥: Position the strap on the wrist.

Wrap around the wrist and insert the strap into the tightening buckle.

Adjust the strap using the self-fastening tab.

Care

Product can be washed in accordance with the conditions shown on this notice and on the label. Remove stays before washing. Replace in the original location before next use. Close the self-fastening tabs before washing. Machine washable at 40°C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

This medical device is a regulated health product with CE marking.

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Schieben Sie die Metallschiene © in die Schienenführung auf der Handinnenfläche ein.