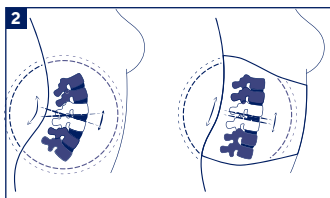
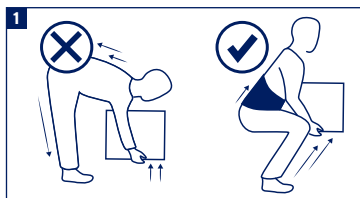
**THUASNE®**

## Lombax vario (T54620)

- (fr) Ceinture de soutien lombaire avec coussin lombaire
- (en) Lumbar support belt with lumbar cushion
- (de) LWS-Stabilisierungsothese mit Pelotte
- (nl) Lumbale rugbrace met lendensteun
- (it) Cintura di sostegno lombare con cuscino lombare
- (es) Faja de sujeción lumbar con almohadilla lumbar
- (pt) Cinta de suporte lombar com almofada lombar
- (da) Rygstøttebælte med lændepude
- (fi) Selkä- ja lantiotukivö lannetyynyllä
- (sv) Ryggstöd för ryggen med kudde för ländryggen
- (no) Ortose med lendepute for stabilisering av korsryggen
- (el) Ζώνη οσφυϊκής υποστήριξης με οσφυϊκό μαξιλάρι
- (cs) Bederní podpurný pás s bederním polštářkem
- (pl) Orteza lędźwiowo-krzyżowa z poduszką lędźwiową
- (lv) Muguras jostas daļas atbalsta josta ar polsterējumu
- (lt) Palaikomasis juosmens diržas su juosmens pagalvėle
- (et) Selja-ja nimmepiirkonna tugivöö nimmepadjaga
- (sl) Oporni ledveni pas z ledveno blazino
- (sk) Podporný driekový pás s odnímateľnou driekovou pelotou
- (hu) Ágyéki fűző deréktámasztó párnával
- (bg) Лумбален колан с лумбална възглавница
- (ro) Centură de susținere lombară cu pernă lombară
- (ru) Опорный поясничный корсет с поясничной подушкой
- (hr) Pojas s jastučićem za potporu lumbalnog dijela
- (zh) 易于调节的带有腰部衬垫的腰部稳定护带
- (ja) 腰椎パッド付き腰サポートベルト
- (ar) حزام لدعم أسفل الظهر مزود بوسادة

| 3            | 4 * |
|--------------|-----|
| 60 - 85 cm   | 1   |
| 86 - 112 cm  | 2   |
| 113 - 140 cm | 3   |

(fr)

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Conserver cette notice.

# LUMBAR SUPPORT BELT WITH LUMBAR CUSHION

## Description

- Elastic fabric ① (polyamide, polyester, elastane, viscose)
- Four reshapable stays ② (brushed tempered steel)
- Two abdominal panels with touch fastening system ③ (polyamide) with independent lower ④ and upper ⑤ straps
- Two flexible stays ⑥ (polyoxymethylene)
- Two finger loops ⑦ (polyester)
- 3D knit back panel (polyester)
- Edging (polyester, elastodiene, latex)
- Additional touch fastening system ⑧ (polyamide)
- Lumbar cushion ⑨ (polyamide, elastane)

## Mechanism of action

The **Lombax vario** belt relieves low back pain, supporting the lumbar spine: compartmental lumbar unloading is therefore achieved via increased intra-abdominal pressure. This device uses back pads, implementing a pulley-like system to distribute pressure.

## Indications

Adult patients with the conditions indicated and compatible with the size chart\*:

- Non-specific low back pain (acute, subacute and chronic phases)
- Lumbar-spinal stenosis

## Destination:

These braces can only be used for the treatment of the back indications listed.

## Fitting instructions

- Choose the appropriate size to fit the patient, referring to the size chart (indicated on the box and on the instruction leaflet).
- Before use, ensure that the product is not damaged.
- Wear the belt on top of a t-shirt or thin vest.
- Open the belt and lay it out completely flat so that you can see all of the components.
- If recommended by your healthcare professional, position the lumbar cushion ⑨ in the belt using the touch fastening system.
- Position the belt around your waist, placing your right hand through the finger loop ⑦ located on the right lower strap ④A. The belt must be centred over the spine. The bottom of the belt must stop at the top of the crease between the buttocks. The Thuasne logo indicates which way up the belt should be put on. The Thuasne name must be the right way up.
- Fasten the belt using the wide touch fastening straps: attach the lower strap ④B on the left side, adjusting to the desired tightness. Repeat the procedure with the upper strap ⑤C.
- If necessary, adjust the tightness to suit your requirements.

## Contraindications

- Do not use on pregnant women.
- Do not use in the event of known allergy to any of the materials (see product composition in the

Description section).

- Do not use in the event of a hiatus hernia.
- Do not use in the event of bone cancer with spinal metastases. Do not use in the event of deficient respiratory functions.

## Precautions

It is recommended that a health professional adjust the stays to the shape of the patient's back before the belt is worn for the first time.

For temporary use, combined with functional rehabilitation if necessary, following the advice or prescription of a health professional.

Consult a health professional if discomfort or pain occurs. Do not use the device if it is damaged.

Do not wear the product while sleeping.

Do not wear the product in a medical imaging machine. Store the device at room temperature, in its original packaging.

Reusable medical device. For hygiene and performance reasons, do not re-use the product for another patient.

Dispose of the product with household waste (no special disposal conditions).

## Unwanted side effects

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

## Washing instructions

- Remove the lumbar cushion and fasten the touch fastening system before washing.
- Machine wash at a maximum temperature of 40°C (forty degrees Celsius), using the delicates programme.
- Do not machine wash the device more than 10 times. Do not dry clean.
- Do not wash the device using aggressive products containing chlorine or oxygenated whitening products.
- Do not tumble dry. Do not iron.
- Squeeze out excess water and dry flat away from heat sources.
- Clean the lumbar cushion with a damp cloth.

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#### **Thuasne Benelux B.V.**
#### **Thuasne Scandinavia AB**
#### **Thuasne Cervitex Ltd**
#### **Thuasne UK Ltd**
#### **Thuasne (Shanghai) Medical Devices Co., Ltd.**