THUASNE®

Cicatrex Contour

de Gebrauchsanweisung - Narbenkompressionsbekleidung
en Instructions for use - Scar compression garment
fr Consignes d'utilisation - Vêtement compressif de cicatrisation
it Istruzioni per l'uso - Indumenti di compressione per cicatrici
es Instrucciones de utilización - Prendas de compresión para cicatrices
pt Instruções de uso - Vestuário de compressão para cicatrizes
nl Gebruiksaanwijzing - Compressiekleiding voor littekens
da Brugsanvisning - Kompressionstøj til arbehandling
sv Bruksanvisning - Ärrkompressionsplagg
fi Käyttöohje - Painevaatteet arpihoitoon
no Bruksanvisning - Arrkompresjonsbekledning
el Οδηγίες χρήσης - Συμπιεστικό ένδυμα για ουλές
cs Návod k použití - Kompresivní prádlo pro terapii jizev
pl Sposób użytkowania - Odzież kompresyjna do blizn
et Kasutusjuhend - Kompressioonriietus armide raviks
lv Lietošanas instrukcija - Rētu kompresijas apģērbi
lt Naudojimo instrukcija - Kompresinė apranga randams gydyti
sk Návod na použitie - Kompresívne odevy na liečbu jaziev
sl Navodila za uporabo - Kompresijska oblačila za brazgotine
hu Használati utasítás - Hegterápiás kompressziós ruházat
ro Instrucțiuni de utilizare - Îmbrăcăminte compresivă pentru cicatrici
bg Инструкция за употреба - Компresионно облекло при белези
hr Upute za uporabu - Kompresijska odjeća za ožiljke
ru Инструкция по использованию - Изделия из компрессионного трикотажа для лечения рубцов
tr Kullanım talimatı - Yara izi kompresyon kıyafeti
ko 독일어 - 흉터 압박복
ja 使用説明書 - 瘢痕治療用圧着衣
ar ملابس ضاغطة للندوب - لامعت سال اتاداشرا

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Purpose

Compression textiles are basic elements in scar tissue therapy. The objective is to obtain flat and elastic scar tissue which does not reduce the quality of life.

With continuously planar pressure, Cicatrex Contour compensates for the resilience and surface elasticity of the skin. Surface elasticity presents the natural limit for scarring to prevent the scar rising above the level of the skin. Blood circulation in the scar will be reduced, collagen fibres are aligned in parallel. Cicatrex Contour is hypoallergenic, prevents inflammations in the tissue, relieves itching and provides comprehensive protection for the fresh scar.

Cicatrex Contour is a compression textile that is used to improve scar/wound healing until the scar has completely healed. It is used to stabilize surgical results, for the medical treatment of lymphoedema and for lipedema pain.

Compression therapy generally is a long-term treatment and requires the corresponding products which – if the medical indication is given – the patient him/herself applies upon the doctor's instruction.

Properties

Cicatrex Contour compression garment combines efficient medical care and high wearing comfort. The material's high elasticity provides for ease of putting the material on or taking it off, and it also ensures great freedom of movement.

Cicatrex Contour compression garment is hypoallergenic, prevents inflammations in the tissue, relieves itching and provides comprehensive protection for the fresh scar.

Cicatrex Contour is a standard compression textile available as leggings with a high waist, leggings with a low waist or as panty girdle with a high waist for the lower extremities. The Cicatrex Contour for the upper part is available as a jacket or waistcoat.

Indications

Damage of the skin and the skin's healing function with hypertrophic scarring or keloid formation.

- Following injuries / burns / surgical interventions
- Skin grafts
- For improvement of scar / wound healing until complete healing of the scar
- For the stabilisation of surgical results
- Medical care of acute, chronic and lymph oedemas
- Lipedema pain

Absolute contraindications

Peripheral arterial occlusive disease (PAOD) stage III/IV; decompensated heart failure; septic phlebitis; phlegmasia coerulea dolens; acute erysipelas; acute bacterial, viral or allergic inflammations; untreated or advanced dermatoses /contact dermatitis; untreated open wounds; swelling of extremities of unknown origin; untreated malignant tumours.

Relative contraindications

Pronounced exudative dermatoses; malignant lymph oedema; peripheral arterial occlusive disease (PAOD) stage I/II; peripheral neuropathy associated with diabetes mellitus; sensitivity disorders of extremities; compensated heart failure; skin infections; dermatoses (e.g. blistering dermatoses); intolerance of pressure or of product ingredients; primary chronic polyarthritis. In case of open wounds in the treatment area, the wound area must first be covered with an adequate wound dressing or a bandage before the compression garment is applied.

Thuasne Deutschland GmbH accepts no liability for failure to observe these contraindications.

Side effects

No side effects are known if properly used. In case of any negative changes during the application, please consult your doctor or the medical retailer immediately. In case of complaints in connection with the product, such as damage of the knitted fabric or lack of fit, please contact directly your medical retailer.

The manufacturer shall not be liable for damages / injuries which resulted due to improper handling or misuse.

Precautions

Follow the instructions given by your doctor, by an orthopaedic technician or medical expert personnel. If you experience severe pain or discomfort, please see your doctor, an orthopaedic technician or medical expert personnel. Store at room temperature, preferably in the original packaging.

To guarantee its effectiveness and for reasons of hygiene, the product may not be passed on to other patients. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Directions for putting the garment on

Your medical retailer will be pleased to show you the correct application. We recommend that compression garments are permanently worn 24 hours per day.

Rough, unfiled fingernails or sharp-edged rings etc. may damage the extremely fine knitted fabric. Therefore wear rubber gloves please. We recommend our fitting aids if there are difficulties putting on the garment. In case of high UV exposure due to sun light, our skin needs to be protected. This is especially true for scar tissue. We therefore recommend to use, under your compression garment, a suitable sunscreen product in the form of creams or the like.

Cicatrex Contour jacket/waistcoat: First slip into one sleeve, then into the other. Pull the jacket/waistcoat over your shoulders and adjust the jacket/ waistcoat. Close the fastening. Ensure that there are no creases and smooth out any creases. Make sure that any breast cups or anatomically shaped garments fit comfortably and without creases.

Cicatrex Contour leggings/panty girdle: Open the fastener, if present. Step into the leggings/panty girdle one after the other and pull them up to the calf. Then use both hands to slowly pull the leggings/ panty girdle up to the waist without creases. Close the fastening, if present. Make sure they are wrinkle-free and smooth out any creases.

Care

Compression garments should be washed daily, if possible. The garment fabric not only collects remainders of ointments and grease but also skin scales, sweat and dust. Regular washing will protect you from infections and your garment from wear. You need a spare set of garments so that you need not interrupt your compression therapy while the other set is washed and dried.

Always turn your compression garment inside out to wash it. Use a mild detergent for washing by hand or for machine wash (at 40° C) without optical brightener and fabric softener. In a laundry bag, compression garments are specially protected. In order to reduce the drying period, put the compression garments into a terry cloth towel, roll it up firmly and press the garment out (do not wring it out). Do not leave the moist compression garment in the wet towel. Hang up your garment loosely and leave it to air-dry. Never put it in the sun or onto a radiator to dry.

Do not expose compression garments to any damaging effects, such as: strong sunlight, intense heat, ointments, oils and greases, stain remover, gasoline and aggressive detergents.

Material composition

Please find the exact information on the sewn-in fabric label.

Note: The sewn-in fabric label is an important component of the compression garment. If you remove this labelling from a Thuasne® product, any claim for warranty, repair or replacement shall be waived.

Material composition of Cicatrex Contour: 59% polyamide, 41% elastane

Storage information and duration of use

Store in a dry place and protect from sunlight. For series products, a maximum shelf life of 36 months applies. This is followed by the duration of use of 6 months for the compression garment. The hourglass pictogram shows the duration of use printed on the carton label. Customised manufacture concerns products for immediate use.

For serial sizes and custom-tailored garments, we guarantee their medical efficacy for a maximum period of wear of six months. Prerequisite for this will be the proper treatment of the compression materials (for example, their care or putting them on or taking them off). If necessary, your doctor should then prescribe new compression garments. In case of any significant change of relevant body sizes (e.g. due to a successful therapy, weight change), new measurements and a new supply may also be justified earlier already.

Disposal

You may dispose of your medical compression garment via the residual or household waste. No special criteria exist for their elimination.

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Thuasne UK Ltd