THUASNE®

SILIMA®

de	Gebrauchsanweisung - Post-OP BH Cara
en	Instructions for use - Post-OP bra Cara
fr	Consignes d'utilisation - Soutien-gorge postopératoire Cara
it	Istruzioni per l'uso - Reggiseno post-operatorio Cara
es	Instrucciones de utilización - Sujetador postoperatorio Cara
pt	Instruções de uso - Soutien pós-operatório Cara
nl	Gebruiksaanwijzing - Post-OP beha Cara
da	Brugsanvisning - Post-OP BH Cara
sv	Bruksanvisning - Post-op behå Cara
fi	Käyttöohje - Cara - postoperatiivinen kompressorintaliivit
no	Bruksanvisning - Post-OP BH Cara
el	Οδηγίες χρήσης - Μετεγχειρητικός στήθόδεσμος Cara
cs	Návod k použití - Pooperační podprsenka Cara
pl	Sposób użytkowania - Biustonosz pooperacyjny Cara
et	Kasutusjuhend - Cara operatsioonijärne rinnahoidja
lv	Lietošanas instrukcija - Pēcoperācijas krūšturis Cara
lt	Naudojimo instrukcija - Pooperacinė liemenėlė „Cara“
sk	Návod na použitie - Pooperačná podprsenka Cara
sl	Navodila za uporabo - Pooperativni nedrček Cara
hu	Használati utasítás - Cara operáció utáni melltartó
ro	Instrucțiuni de utilizare - Sutien post-operator Cara
bg	Инструкция за употреба - Следоперативен сутиен Cara
hr	Upute za uporabu - Postoperativni grudnjak Cara
mk	Упатства за користење - Post-OP BH Cara
sr	Uputstvo za upotrebu - Postoperativni grudnjak Cara
tr	Kullanım talimatı: Ameliyat Sonrası Sutyen Cara
ru	Инструкция по использованию - Послеоперационный бюстгальтер Cara
ja	使用説明書 - 術後ブラジャー Cara
ar	(Post-OP BH Cara) تعليمات استخدام حمالة الصدر بعد الجراحة
he	הוראות שימוש חזיית Cara לאחר ניתוח
ko	독일어: 사용설명서 - Cara 수술 후 브라
zh	使用说明书 - Post-OP BH Cara

de

Post-OP BH Cara

Postoperativer Kompressions-BH (Art.-Nr. T57230)

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en

Cara postoperative bra

Postoperative compression bra (Art. No. T57230)

Indication and purpose

The postoperative compression bra is used for uniform compression in the mammary area. Haematomas and postoperative oedemas are minimised. The bra's comfortable handling makes the inspection of wounds easier and reduces pain. Stabilisation and immobilisation of the post-surgical breast is realised after any plastic breast surgery.

The postoperative compression bra consists of a highly elastic and skin-friendly material. The material is doubled in the chest-enclosing front and thus ensures uniform compression. Triple adjustable straps, hook and eye closures, as well as the front zipper ensure that the bra is easy to adjust and to put on or take off.

The broad, elastic underbust band prevents rolling, pressure and abrasion points, as well as constrictions. The bra is offered in many different sizes. In combination with the postoperative belt (Art. T57231), the bra can be used for the fixation and stabilisation of the breast implant after all plastic breast surgeries.

Indications

Ensuring results after surgical interventions involving:

- Conservative breast surgery
- Breast reconstruction
- Breast reduction
- Breast augmentation

Contraindications

Allergic or inflammatory skin changes (e.g. swelling, reddening of the body area to be cared for). Use only on intact skin, not on infected or acutely inflammatory skin. Please consult with the treating physician regarding the indication and wearing mode of this orthopedic aid.

Sizes

	AA/A B/C D/E	AA/A B/C D/E	AA/A B/C D/E	AA/A B/C D/E	AA/A B/C D/E	AA/A B/C D/E	AA/A B/C D/E	AA/A B/C D/E*	AA/A* B/C*	AA/A* B/C*	AA/A* B/C*
Int/D	70	75	80	85	90	95	100	105	110	115	120
F/E	85	90	95	100	105	110	115	120	125	130	135
I	1	2	3	4	5	6	7	8	9	10	11
GB/US	32	34	36	38	40	42	44	46	48	50	52
AUS/NZ	10	12	14	16	18	20	22	24	26	28	30

*Available only in black colour

How to put on/strap on

The bra is put on and closed in front with a hook and eye closure and the zipper. The length of the straps is triple adjustable.

Precautionary measures

Please follow the instructions by the physician or the orthopedic technician. In case of severe discomfort or unpleasant sensations, please contact your physician or orthopedic technician. Store at room temperature, preferably in its original packaging. Please follow the instructions by the physician or the orthopedic technician. This medical device is intended for the care of an individual female patient. In order to ensure its efficacy and for reasons of hygiene, this device may not be passed on to other female patients. Any serious incidents which occurred in connection with this product are to be reported to the manufacturer and the responsible authority of the member state in which the user and/or the patient are resident.

Washing instructions

60 degrees water temperature; no fabric softener (also see textile label)
Prior to washing, please close all closures!

Warranty

According to the statutory provisions, warranty is extended to material and processing if properly handled according to the instructions for care.

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Thuasne Scandinavia AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
Thuasne Deutschland GmbH