THUASNE

fr	Chevillère élastique	4
en	Elastic ankle support	5
de	Elastische Knöchelbandage	6
nl	Elastische enkelbrace	8
it	Cavigliera elastica	9

	1	19 - 21 cm
	2	21 - 23 cm
	3	23 - 25 cm
	4	25 - 27 cm
	5	27 - 30 cm

fr	Tour de cheville
en	Ankle circumference
de	Knöchelumfang
nl	Omvang van de enkel
it	Giro caviglia

fr	Maintien	Compression	Effet proprioceptif
en	Support	Compression	Proprioceptive effect
de	Halt	Kompression	Propriozeptive Wirkung
nl	Fixatie	Compressie	Proprioceptieve werking
it	Sostegno	Compressione	Effetto propriocettivo

MD

fr	Dispositif médical
en	Medical Device
de	Medizinprodukt
nl	Medisch hulpmiddel
it	Dispositivo medico

fr	Un seul patient - à usage multiple
en	Single patient - multiple use
de	Einzelner Patient - mehrfach anwendbar
nl	Eén patiënt - meervoudig gebruik
it	Singolo paziente - uso multiplo

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ELASTIC ANKLE SUPPORT

Description/Destination

This device is a proprioceptive elastic ankle brace providing joint support.
The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Polyamide - cotton - elastane.

Properties/Mode of action

Elastic compressive knitting.
Comfort zone over the ankle bones.

Indications

Post-traumatic residual pain and/or edema.
Proprioceptive support for rehabilitation.
Sport activities resumption.

Contraindications

Do not apply the product in direct contact with broken skin.
Do not use in the event of known allergy to any of the components.
History of venous or lymphatic disorders.

Precautions

Verify the product's integrity before every use.
Do not use the device if it is damaged.
Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional:

- in case of doubt before use,
- in case of uncertain diagnosis

Consult a healthcare professional and stop using the device:

- in case of discomfort, significant hindrance, pain, abnormal sensations.
- in case of changes in the device's performance.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

Pull the ankle brace over the ankle joint, like a sock.

Place the malleoli correctly, on the comfort zones.

Care

Product can be washed in accordance with the conditions shown on this notice and on the label. Machine washable at 40°C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

This medical device is a regulated health product with CE marking.

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©Thuasne - 2024901 (2021-04)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186 – capital 1 950 000 euros